INSTRUCTIONS FOR USE: Liquid Scope HelpOR™

Endoskop/laparoskope warmer and scope cleaning device

EN

INDICATIONS FOR USE / CLINICAL BENEFITS

The Liquid Scope HelpOR is to be used before and during endoscopic and laparoscopic surgery to prevent condensation and fogging of the scope and keep the scope lens and trocar valves free of debris.

CONTRAINDICATIONS

This device is not intended for use except as indicated.

CAUTIONS / WARNINGS

All users must read and be familiar with all instructions, warnings, contraindications and precautions in this package insert before using the device.

- For single patient use only. The design of this device may not perform as intended by the manufacturer if it is reused. The manufacturer cannot guarantee the performance, safety and reliability of a reprocessed device.
- Device is sterile if the package is dry, unopened and undamaged. DO NOT use if the pouch is damaged or the packaged seal is broken.
- Do not re-sterilize the device. Re-sterilization may compromise the integrity of the device and create the risk of contamination and unintended patient infection.
- Verify compatibility. Inspect the endoscope/laparoskope and trocar for sharp edges that could interfere with the components of the Liquid Scope HelpOR (liquid scope warmer, microfiber cloth and trocar swabs).
- Improper use of the Liquid Scope HelpOR may lead to patient injuries and/or damage of the endoscope/laparoskope and trocar.

Liquid Scope WarmOR™

- Device is designed for short term use. DO NOT use for more than 24 hours.
- Take care when placing the endoscope into the Liquid Scope WarmOR
- DO NOT use if punctured. Exposure to the sodium acetate solution may cause irritation to the eyes and skin. Immediately rinse with water.
- DO NOT insert endoscope/laparoskope after the device has been activated and changed its consistency (turned rigid).
- Peak temperature (approximate): 50 +/- 5°C.

Scope CleanOR™

- Trocar swabs should not be inserted on an angle or extended beyond the distal end of the trocar.

INSTRUCTIONS FOR USE




Liquid Scope WarmOR™

- Remove the Liquid Scope WarmOR™ from its sterile packaging (Figure 1 or 2). Inspect the device for damage or contamination, and discard if any is noted. Otherwise, place on sterile field.
- Place the distal tip of the endoscope / laparoskope in the middle of the Liquid Scope WarmOR and align it to the centre fold of the Liquid Scope WarmOR (Figure 3, Step #1). Close the buttons of the device (Figure 3, Step #2).
- Locate and flex the metal disk inside the fluid pack 2 or 3 times to start the warming (Figure 3, Step #3).
- Allow 3 to 5 minutes to warm (Figure 3, Step #4). The Liquid Scope WarmOR will reach its peak temperature in 5 minutes. Note: For best result, activate the Liquid Scope WarmOR about 5 to 7 minutes before surgery. Wrap the device and the endoscope /laparoskope in a double layer sterile towel to prolong the heating period for approximately 45min up to one hour.
- Carefully remove the endoscope / laparoskope from the Liquid Scope WarmOR ready for use (Figure 3, Step #5).
- Upon completion of the procedure, the Liquid Scope WarmOR should be discarded in biohazardous waste as per standard procedure.

Scope CleanOR™

- Using sterile technique, remove the Scope WipOR microfiber cloth and the two Trocar CleanOR swabs from the packaging. Inspect the device for the damage or contamination, and discard if any is noted. Otherwise, place it on sterile field. During the procedure, anytime the scope is removed from the body:
- Clean the Scope lens using the Scope WipOR microfiber cloth.
- To remove debris from the trocar valves, use the small trocar swab for 5-8mm trocars, and the larger trocar swab for 10-12mm trocars (Figure 4). NOTE: Trocar must be fully inserted into the abdominal cavity prior to use of the Trocar CleanOR.
- Select the appropriate trocar swab and insert its foam head into the trocar valve. Advance the swab into the valve, but do not extend the foam head beyond the distal end of the trocar. Move the swab back and forth to absorb any fluid and clean debris from the cannula. Do not release the trocar swab. NOTE: The shafts of the trocar swab are radiopaque. Trocar CleanOR must be used under direct vision.
- Check the cannula for free passage. If this is not the case, repeat the process.
- Before disposing the trocar swabs and microfiber cloth visually inspect the devices to ensure they are intact and all components have been retrieved.
- Upon completion of the procedure, the Scope CleanOR should be discarded in biohazardous waste as per standard procedure.

Note: Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

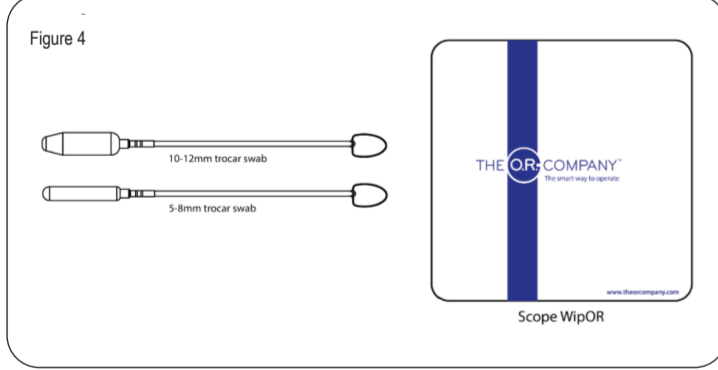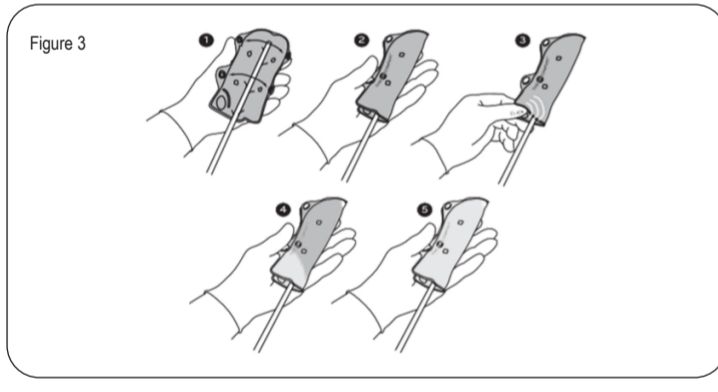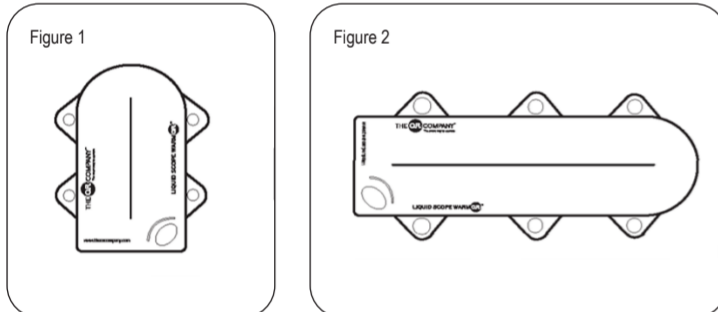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