

## Instructions for use

### Instruments Guards

Includes all products except for Web Guards #09-9-001 through 09-9-007

#### Intended Use:

The Instrument Guards are intended to protect surgical instruments through handling and sterilization.

#### Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Instructions for Use:

- Place guard over clean instrument.
- Guards maybe left on during handling, storage, and sterilization by steam or ethylene oxide.
- Flash steam sterilize for 4 minutes at 132° C (270° F).
- Conventional steam sterilize for 30 minutes at 121° C (250° F).

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

