MEGADYNE™

Megadyne Medical Products Inc
11506 South State Street
Draper, Utah 84020 USA
+1(801)576-9669

96007

Federal (USA) law restricts this device to sale by or on the order of a Licensed Healthcare Practitioner.

Olsen Medical, LLC
3230 Commerce Center Place
Louisville, Kentucky, 40211 USA
(502) 813-5022

Symmetry Surgical GmbH
Maybachstraße 10, 78532
Tuttlingen, Germany
Phone: +49 7461 96490
Fax: +49 7461 77921

96007 Monopolar Adapter
96007 Monopolær Adapter
Monopolaire adapter 96007
Adaptateur monopolaire 96007
Monopolarer Adapter 96007
Μονοπολικός προσαρμογέας 96007
Adattatore monopolare 96007
96007 モノポーラ・アダプタ
Adaptador monopolar 96007
Adaptador Monopolar 96007
96007 Monopolär adapter
96007 – Monopolaarinen sovitin
96007 Monopoläris adapter
96007 Monopolární adaptér

96007 Monopolar Adaptör
96007 Monopolar adapter
96007 Monopolárny adapter
Adapter monopolar 96007
96007 Монополярный адаптер
Jednobiegunowy adapter 96007
96007 单极适配器
96007 모노폴라 어댑터
Монополярен адаптер 96007
Monopolarni prilagodnik 96007
96007 monopolaarne adapter
96007 Monopolärs adapteris
96007 vienpolis adapteris
Enopolni vmesnik 96007

ENGLISH

Description:

The 96007 adapter is a non-sterile, reusable accessory for use with most disposable and reusable foot switching electrosurgical devices including electrosurgical pencils, suction coagulators, and monopolar forceps. Each reusable device is designed to withstand a maximum of 20 sterilization cycles when properly cared for and sterilized according to the instructions below.

Indications:

Megadyne's 96007 Monopolar Adapter is designed to bring high frequency electrical power from the electrosurgical generator to the active electrode and are intended for use with a maximum voltage of 5000 Volts.

Handling:

To insert wire into the adapter:

1. Squeeze the two flanges together using thumb and forefinger.
2. Insert small brass terminal fully into hole of adapter.
3. Release flanges – wire will lock in place.
4. Test by pulling on wire. A properly inserted wire will not pull out.

To release wire from the adapter:

1. Squeeze the two flanges together using thumb and forefinger.
2. Jiggle and slightly twist wire while removing from adapter.

Cleaning:

1. Prepare a proteolytic enzyme solution according to the manufacturer's instructions.
2. Completely submerge adapter in the enzyme solution and gently shake to remove trapped bubbles. Allow to soak for a minimum of 10 minutes.
3. Remove the instruments from the enzyme solution and rinse in tap water for a minimum of one (1) minute.
4. Place instruments in a suitable validated washer/disinfector. Follow the washer/disinfector manufacturer's instructions for loading the instruments for maximum cleaning exposure. The following minimum wash cycle parameters are recommended:

Cycle	Description	Time
1	Pre-wash --- Cold Softened Tap Water	2 minutes
2	Enzyme Spray & Soak --- Hot Softened Tap Water	1 minute
3	Rinse ---- Cold Softened Tap Water	
4	Detergent Wash ---- Hot Tap Water (64-66°C/146-150°F)	2 minutes
5	Rinse ---- Hot Purified Water (64-66°C/146-150°F)	1 minutes
6	Hot Air Dry (116°C/240°F)	7 to 30 minutes

Storage:

Adapter must be stored in a clean, dry, moisture free area with suitable protection from extreme temperature and humidity.

Sterility:

1. Megadyne's 96007 Monopolar Adapter is sold NON-STERILE. Steam sterilize before each use per the instructions below.
2. Arrange all devices to allow access of steam to all surfaces. Open hinged devices and ensure devices are disassembled if it is recommended.

Sterilization Cycle	Temperature	Time	Minimum Dry Time	Device Configuration
Prevacuum Flash	270°F (132°C)	3 Minutes	–	Sterilization Tray
Prevacuum	270°F (132°C)	4 Minutes	20 Minutes	Wrapped/Individually Pouched
Gravity Displacement	270°F (132°C)	10 Minutes	10 Minutes	Wrapped

Caution:

- Inspect every device before each use. Damaged electrosurgical devices may cause electrical burns to the patient or physician.
- Consult RF generator Owner's Manual. Use only in the active RF generator output connections.
- Do not perform electrosurgery in the presence of flammable anesthetics or other flammable gases, fluids or objects, or in the presence of oxidizing agents, as fire could result.
- Do not bend, cut, or modify any electrosurgical device before or during surgery.
- Establish a protocol for device examination and maintenance.

Contraindications:

The device is not intended to be used except as indicated.

Intended Users:

Trained clinical and medical Personnel.

Disposal:

Biohazard- Dispose of the device according to the facility procedures.

Warning:

Improper use of any element in electrosurgical systems may cause serious burn, shock, and fire or explosion hazard. The use of the device with safely functioning and compatible handles, cords, electrosurgical generators and accessories is the user's responsibility. Read and understand all precautions and instructions before attempting to use any electrosurgical device.

Warranty:

All products are warranted to be free from defects in workmanship and materials. Any product that proves defective in workmanship or materials will be repaired or replaced at our discretion free of charge. Repair, modification, or alteration to any product that is performed by anyone not authorized by Megadyne will result in the immediate loss of warranty.

EU Commission Statement:

European Commission requires that any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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