# Symmetry Surgical® Laparoscopic Instruments

## Symmetry Surgical Inc.

3034 Owen Drive,  
Antioch, TN 37013 USA

☎ 1-800-251-3000

Fax: 1-615-964-5566

[www.symmetrysurgical.com](http://www.symmetrysurgical.com)

Symmetry Surgical GmbH  
Maybachstraße 10  
78532 Tuttlingen, Germany

☎ +49 7461 96490

Fax: +49 7461 77921

# CE 0086

(These instructions pertain to  
both Class I and Class II devices)

LCN 204198-001/H

© 2004–2015 Symmetry Surgical

♻ Revised 09/15

# ENGLISH

## IMPORTANT INFORMATION

Please Read Before Use

## Symmetry Surgical® Laparoscopic Instruments

### Product Description

These instruments are designed for use in minimally invasive surgery through a cannula of 5.5 mm or 10.5 mm inner diameter. They are available in working lengths of 330 mm, 450 mm, or 500 mm. Please see the product label for description of contents.

These instruments are available in one-piece construction or two-piece construction with detachable handles and shafts. See instructions below for handle and shaft assembly and disassembly if using a two-piece construction.

### Precautions

Thoroughly read the instructions for use that accompanied the electro-surgical generator before using electro-surgical laparoscopic instruments.

This instrument is provided nonsterile and must be cleaned and sterilized before use, including initial use.

Delicate instruments, such as this device, must be handled with care, must only be used for their intended purpose, and must be protected against damage during reprocessing.

### Instructions for Assembly and Disassembly in Two-piece Construction (see Figure 1)

To assemble instruments with detachable handles:

1. Disengage the ratchet mechanism and spread the rings of the handle.

- 
2. Pull the collar on the shaft backwards to expose the ball end of the shaft.
  3. Drop the ball into the slot visible on the handle.
  4. Slide the collar onto the shaft to meet the rotating knob on the handle.
  5. Secure the shaft to the handle by turning the collar.

**Note:** When the connection is secure, you will be able to rotate the shaft 360 degrees by turning the rotating knob.

To disassemble the instrument:

1. Loosen the collar on the shaft.
2. Disengage the ratchet mechanism and spread the rings of the handle.
3. Align the ball on the shaft with the slot on the handle.
4. Lift the shaft from the handle.

## Reprocessing Instructions

Refer to Symmetry Surgical Instructions for Use #204233 for updated information regarding reprocessing of both one-piece and two-piece constructions.

Copies of this instruction sheet are available by requesting IFU #204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000

Fax: +1 615 964 5566

E-mail:

[ssi-customerservice@symmetrysurgical.com](mailto:ssi-customerservice@symmetrysurgical.com)

## Service and Repair

For service or repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical Inc.  
3034 Owen Drive,  
Antioch, TN 37013 USA

Always include a repair purchase order number and a written description of the problem.

## Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship for one (1) year from the date of purchase. Use of this device outside of its intended purpose shall be considered misuse and shall cause this warranty to become void. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

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