Prone Face Pillow

Non-Sterile

Prone Face Pillow 714100

Aspen Surgical Products, Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316 USA
Phone +1 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

EMERGO REP
Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

For Country of Origin See Product Label

English

Intended Use

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User/Patient Target Groups

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instruction for Use:

- Allow proper expansion of prone pillow prior to use.
- Carefully place pillow over patients face in the supine position after intubation.
- Hold pillow in place while patient is being moved to the prone position.
- Properly align spine in prone position.
- Check tube placement after patient is in the prone position.
- Device is intended for limited contact duration, less than or equal to 12 hours.
- Discard after use.

Precautions:

- Ensure that patients face is centered on the pillow.
- Ensure that the prone face pillow is facing up and that the patients face in on the soft foam.
- Ensure that intubation tube is secure in the premade holder.
- Frequently check patient neck, head, eyes, nose and mouth to ensure that a safe position is maintained and offload as required per facility protocol.
- This device is single use only.
- Do not sterilize.

Warning

Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.

Disposal

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional and/or local laws and regulations.

Notice to Users and/or Patients in EU

Any serious incident that has occurred in relation to the device should be reported to Aspen Surgical Inc., the governing body, and the Competent authority of the member state in which the incident occurred.

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Emergo Consulting (UK) Limited