## EN Instructions for Use Reusable Grounding Pad Cable

**WARNING: Inspect prior to use. Do not use if cable insulation is cracked, split, or otherwise damaged. Electric shock could result.**

**CAUTION:** Do not autoclave. Damage may occur.

**CAUTION:** Do not use a reusable (steel) grounding plate with A1202C or A1252C cables. Breakage of the clip may occur.

**Use:** Grounding pad cables are used to create a pathway to direct the electrical current through the patient back to the generator. Cables are only compatible with specific generators and grounding pads.

**To Clean:** Wipe with a cloth or gauze that has been soaked in warm water and dry with a clean cloth. Disinfect using a phenol or glutaraldehyde based solution prepared according to the manufacturer’s recommendations. Wipe again with warm water and dry using a clean cloth.

**CAUTION:** Do not soak the cable in any disinfection solution as damage may occur.

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**STORAGE**  
10 to 75 % non-condensing

**OPERATE**  
30 to 75 % non-condensing

**TRANSPORT**  
10 to 100 % including condensation

**30° C STORAGE**  
**40° C OPERATE**  
**47° C TRANSPORT**

10° C  
10° C  
-40° C

**MC-18250 Rev. 9**  
**2023-06-30**

Cable Catalog Number	Grounding Pad	Generator	Use
A1202C	A1202 (Single-Use)	A950	Open black clip and insert grounding pad. Close clip by pressing downward until the lid clicks into place. Do not force the clip to close.
A1252C	ESRS, ESRE (Single-Use)	A952 A1250 series A2250, A2350 A3250, A3350 IDS-200, IDS-210 IDS-300, IDS-310	
A1204C	All reusable (steel) grounding plates	A950	Align female end of cable with male end of grounding plate. While holding the cable, turn the plate clockwise until the plate is securely attached.
A1254C	All reusable (steel) grounding plates	A952 A1250 series A2250, A2350 A3250, A3350 IDS-200, IDS-210 IDS-300, IDS-310	

**To Clean:** Wipe with a cloth or gauze that has been soaked in warm water and dry with a clean cloth. Disinfect using a phenol or glutaraldehyde based solution prepared according to the manufacturer’s recommendations. Wipe again with warm water and dry using a clean cloth.

**CAUTION:** Do not soak the cable in any disinfection solution as damage may occur.

**CAUTION:** Do not use a reusable (steel) grounding plate with A1202C or A1252C cables. Breakage of the clip may occur.

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