



Instructions For Use

Protek™ PullUp™ Form Fitted Probe Cover

aspensurgical.

INTENDED PURPOSE / INTENDED USE

This is a medical device used over imaging probes or instruments to provide a sterile protection, and to minimize cross contamination during diagnostic imaging procedures.

STERILE EO



Do not use if sterile package is damaged.

USE INSTRUCTIONS

Prepare probe per system's operational manual.

1. Squeeze sides to open cover.
2. Insert probe into cover.
3. Pull cover over probe and cord. Use clips to fasten cover.



Do not re-use.

IMPORTANT! When using probe with trocar, use a trocar that is 2 mm larger than probe, and:

4. Lubricate exterior of cover and trocar seal.
5. To avoid tearing cover, probe must be inserted or removed in a non-angled position to trocar.



This product is single-use only.

Do not reuse. Do not resterilize. Reuse, reprocessing or resterilization of the single use devices may create a risk of cross contamination, patient infection, and/or malfunction of the device.

After use, dispose of single-use components as infectious waste.

Do not use if sterile package is damaged.

For translations of this instructions, go to aspensurgical.com/ifu



Aspen Surgical Products, Inc.
6945 Southbelt Dr, Caledonia, MI 49316
phone +1 616.698.7100 toll-free +1 888.364.7004
aspensurgical.com

EC REP

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

CE
2797

Made in Mexico
1-041-2016 Rev 2