

Heel Protector Pair

HP100

Case Pack: 36 pair



- Pair of single patient use convoluted foam heel protectors.
- Soft foam with appropriate density provides support and reduce pressure to the patient.
- Soft, wide foam straps for ease of use and patient safety.

Not made with natural rubber latex

Instructions for Use

For use on patient on operation room table, bed or stretcher. **Do not allow** patient to walk with HP100 on due to risk of fall.

Follow facility protocol for skin assessment prior to use.

Remove the HP100 from packaging and place the patient's heel in the anatomically oriented heel portion of the cushion. Place the soft Velcro strap across the patient's foot and affix it on the opposite side of the cushion to ensure a secure fit.

Discard after use.


PRIME
MEDICAL llc