INSTRUCTIONS FOR USE

Bard Parker®

aspensurgical.

Conventional Disposable Scalpel

STERILE EO R_x ONLY MD

EC REP

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

For Country of Origin See Product Label

8365998 | DGP73 | REV 1 | 08JAN2024

CE
2797

Aspen Surgical Products, Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316 USA
Phone +1 616-698-7100
Toll-Free 888-364-7004

English

Bard-Parker® Conventional Disposable Scalpel

Intended Use

Intended for tissue separation and other procedures that require a sharp surgical blade to puncture or cut.

Intended User/Patient Target Groups

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instructions for Use:

- Ruler shown on scalpel handle should be used as an indicator only, and not for accurate measurements.
- During use avoid twisting, bending or putting excessive force or strain on the blade in order to help prevent breakage.
- If blade becomes dull or breaks, dispose and replace product.
- A singular blade is intended to cut tissue up to 24 inches.
- Do not use product if product is damaged.

Warning

Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.

Disposal

Dispose in approved puncture-resistant sharps container according to facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Notice to Users and/or Patients in EU

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Emergo Consulting (UK) Limited