For product information, contact:

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Tilberkerd i USA

ONLY

Reusable ElectroSurgical Devices

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Caution:

- Inspect every device before each use. Damaged electroSurgical devices may cause electrical burns to the patient or physician.
- Active electroSurgical devices must be placed away from the patient when not in use. A hoister is recommended.
- Use a wet sponge to remove eschar build up from the forceps tips.
- Consult RF generator Owner's Manual. Use only in the active RF generator output connections.
- Electromagnetic compatibility (EMC) – Olsen Medical devices are considered medical electrical equipment. Medical electrical equipment may require precautions regarding EMC and may need to be installed or put into service according to the EMC information provided in the operator's manual of the generator.
- Do not use electroSurgical equipment unless properly trained to use it in the specific procedure being undertaken. Use by physicians without such training may result in serious, unintended patient injury.
- Do not perform electroSurgery in the presence of flammable anesthetics or other flammable gases, fluids or objects, or in the presence of oxidizing agents, as fire could result.
- Avoid unnecessary or prolonged activation. Activate only when touching or immediately next to the intended point of contact.
- Non-insulated switching contacts of hand controlled forceps must not come in close proximity to or touch the skin or bodily tissue to prevent a burn to the patient or physician.
- Electrode must be compatible with the pen/pencil or handle and ensure that the insulated sleeve fits securely inside the handle. Improperly seated electrode may cause burn or injury.
- Do not bend, cut, or modify any electroSurgical device before or during surgery.
- Keep the voltage/power settings at the lowest possible level to achieve the desired effect.
- Establish a protocol for device examination and maintenance.

Warning:

Improper use of any element in electroSurgical systems may cause serious burn. The use of the device with safety functioning and compatible handles, cords, electroSurgical generators and accessories is the user's responsibility. Read and understand all precautions and instructions before using any electroSurgical device.

Warranty:

All products are warranted to be free from defects in workmanship and materials when used for its intended purpose. Any product that proves defective in workmanship or materials will be repaired or replaced at our discretion free of charge. Repair, modification, or alteration to any product that is performed by anyone not authorized by Olsen Medical® will result in the immediate loss of warranty.

OLSEN® is a registered trademark of Olsen Medical.

Sterilisation Cycle	Temperatura	Tempo	Tempo minimo in secca	Configurazione dispositivo	Tray per la sterilizzazione			
Prevacuum Flash	270°F (132°C)	3 Minutes	–					
Prevacuum	270°F (132°C)	4 Minutes	20 Minutes	Wrappato/ Individualmente sigillato				
Gravità/Displacement	270°F (132°C)	10 Minutes	10 Minutes	Wrappato				

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