Symmetry Surgical® I.M.A. Forceps

Symmetry Surgical Inc.
3034 Owen Drive,
Antioch, TN 37013 USA
☎ 1-800-251-3000
Fax: 1-615-964-5566
wwwu u

Symmetry Surgical GmbH
Maybachstraße 10
78532 Tuttlingen, Germany
☎ +49 7461 96490
Fax: +49 7461 77921

LCN 800-1851-85/M

© 1999–2015 Symmetry Surgical
♻ Revised 06/1

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

Symmetry Surgical® I.M.A. Forceps

Indications

The I.M.A. Forceps are for use during revascularization procedures involving the internal mammary artery.

Catalog No.

80-1851	Symmetry Surgical I.M.A. Forceps, Straight, 7 in. (17.8 cm)
80-1852	Symmetry Surgical I.M.A. Forceps, Bayonet, 8½ in. (21.6 cm)
80-1853	Symmetry Surgical I.M.A. Forceps, Straight, 7 in. (17.8 cm)
80-1857	Symmetry Surgical I.M.A. Forceps, Bayonet, 8½ in. (21.6 cm)
80-1858	Symmetry Surgical I.M.A. Forceps, Straight, 8½ in. (21.6 cm)
80-1859	Symmetry Surgical I.M.A. Forceps, Straight, 8½ in. (21.6 cm)

Inspection

Inspect each instrument before and after use to detect imperfections. Do not use damaged instruments; send them for repair or replacement.

CAUTIONS

The micro mouse tooth tips of the forceps are extremely delicate and must be protected at all times. Use of the I.M.A. Instrument Case (catalog no. 80-1865) for storage is recommended.

Exercise care when cleaning the instruments. Gently clean micro mouse tooth tips. NEVER USE ANY TYPE OF ABRASIVE CLEANER ON THE INSTRUMENTS.

Reprocessing Instructions

Refer to Symmetry Surgical Instructions for Use #204233 for updated information regarding reprocessing the instrument.

Copies of this instruction sheet are available by requesting IFU #204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000
Fax: +1 615 964 5566
E-mail: ssi-customerservice@
symmetrysurgical.com

Service and Repair

For service or repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical
3034 Owen Drive,
Antioch, TN 37013 USA

Always include a repair purchase order number and a written description of the problem.

Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

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