

Instructions for use

Lewiston Hip Positioner Set 712100

Previously Beatty® reorder # 072917

Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:


- For use with Arthrex® Hip Distraction System AR-6529S.
- Ensure perineal pad is secure and covering the positioning device and that it stays in place while the patient is on the table.
- Ensure perineal pad is centered between patient's legs.
- Ensure boot liner is securely fastened around foot before placing in the positioning device.
- Avoid redundant boot material that may wrinkle or cause excess pressure.
- Use on recommended tables only.
- This device is single use only.
- Do not sterilize.

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Place perineal post pad onto positioning device securely.
2. Place patient's foot in liner.
3. Using the hook and loop closure, secure position 1 to 2, then 3 to 4 and lastly 5.
4. Secure foot in positioning device.
5. Secure boot to table using manufacturers recommended instructions.
6. Discard after use.

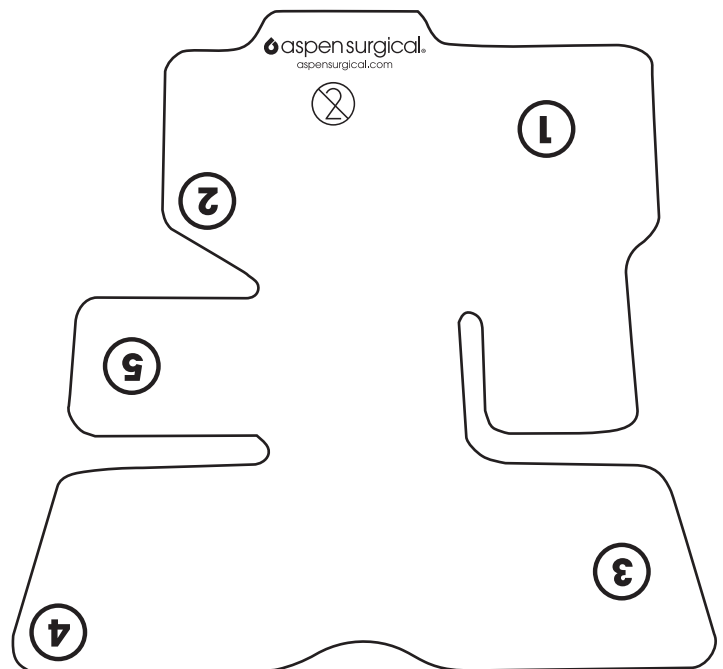

Arthrex® is a registered trademark of Arthrex, Inc. ("Arthrex"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Arthrex.