# KARLIN

## Micro Knife Handle

---

**Symmetry Surgical Inc.**  
3034 Owen Drive  
Antioch, TN 37013 USA  
☎ 1-800-251-3000  
Fax: 1-615-964-5566  
[www.symmetrysurgical.com](http://www.symmetrysurgical.com)

Symmetry Surgical GmbH  
Maybachstraße 10  
78532 Tuttlingen, Germany  
☎ +49 7461 96490  
Fax: +49 7461 77921

**LCN 170415-001/L**  
© 2002–2015 Symmetry Surgical  
♻ Revised 06/15

## ENGLISH

### IMPORTANT INFORMATION

Please Read Before Use

### KARLIN™ Micro Knife Handle

#### Indications

The KARLIN™ Micro Knife Handle, catalog no. 46-3174, is specifically designed and constructed to be used with KARLIN™ disposable blades for hand-held microdissection. The handle is supplied nonsterile and must be sterilized before use, including initial use.

#### WARNINGS

Dull instruments can cause injury, even death.

If using the micro knife, ensure that the blade is stationary.

#### CAUTIONS

Inspect each instrument before using; maintain in a clean and sharp condition.

The instrument's working tip is extremely delicate; do not apply undue force.

Do not strike the instrument.

Do not allow other objects to strike the instrument.

Store the instrument in its case when not in use to minimize potential damage to the instrument.

Do not clean the instrument with alkalines or bleaches.

Always lubricate the instrument handle with a water-soluble lubricant before sterilizing the instrument.

Do not place heavy equipment on top of the instrument.

With the micro knife, do not use pliers or other mechanical assistance to attach the blade to the handle.

#### Assembly

To insert a micro knife blade (catalog no. 46-3177 or 46-3178) into the handle, turn the collet knob counterclockwise. Insert the desired blade as far as you require. Tighten the collet knob by turning it clockwise.

**CAUTION:** Do not use pliers or other mechanical assistance.

**WARNING:** Ensure the blade is stationary before using.

#### Reprocessing

Refer to Symmetry Surgical Instructions for Use 204233 for updated information regarding reprocessing the instrument.

Copies of this instruction sheet are available by requesting IFU 204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000  
Fax: +1 615 964 5566

E-mail:  
ssi-customerservice@symmetrysurgical.com

#### Service and Repair

For service or repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send all instruments for service and repair to:

Symmetry Surgical  
3034 Owen Drive,  
Antioch, TN 37013 USA

Always include the purchase order number and a written description of the problem.

## Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

KARLIN™ is a trademark of Karlin Technology, Inc.

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