CAUTERY

WARNINGS:

DO NOT USE IN THE PRESENCE OF FLAMMABLE GASES/MATERIALS OR IN OXYGEN RICH ENVIRONMENTS. FIRE COULD RESULT.

CAUTERY PRODUCES HEAT IN EXCESS OF 1000 DEGREES F, WHICH CAN CAUSE BURNS OR FIRE FROM MISUSE.

ACCIDENTAL EXPOSURE TO CAUTERY TIP MAY LEAD TO INADVERTENT BURNS TO PATIENT OR USER.

PROLONGED TISSUE EXPOSURE TO CAUTERY MAY LEAD TO DAMAGE OF ADJACENT TISSUE.

CAUTERY TIP WILL REMAIN HOT IMMEDIATELY AFTER DEACTIVATION AND MAY CAUSE INADVERTENT BURNS TO USER OR PATIENT.

THIS MEDICAL DEVICE CANNOT BE EFFECTIVELY CLEANED AND/OR STERILIZED BY THE USER AND THEREFORE CANNOT BE SAFELY REUSED. IT IS INTENDED FOR SINGLE USE ONLY. ANY ATTEMPT BY THE USER TO CLEAN AND RESTERILIZE THIS DEVICE MAY RESULT IN BIOINCOMPATIBILITY, INFECTION, OR OTHER RISKS OF DEVICE FAILURE TO THE PATIENT.

ALWAYS RETAIN THE COVER CAP AND REPLACE IT ON THE CAUTERY WHEN NOT IN USE.

DEACTIVATE THE CAUTERY BEFORE DISCARDING THE DEVICE IN A BIOHAZARD OR “SHARPS” DISPOSAL CONTAINER. FIRST BREAK OR CUT OFF THE CAUTERY WIRE TIP USING HEMOSTATS AND REPLACE THE COVER CAP. REMOVE BATTERIES BY BREAKING OFF END CAP. FAILURE TO FOLLOW THIS INSTRUCTION COULD RESULT IN UNINTENDED ACTIVATION OF THE CAUTERY AND MAY CAUSE A FIRE IN THE DISPOSAL CONTAINER.

WARNING: THIS PRODUCT IS STERILIZED UTILIZING ETHYLENE OXIDE. THESE PRODUCTS CAN EXPOSE YOU TO CHEMICALS INCLUDING ETHYLENE OXIDE (ETO), WHICH IS KNOWN TO THE STATE OF CALIFORNIA TO CAUSE CANCER AND/OR BIRTH DEFECTS OR OTHER REPRODUCTIVE HARM. FOR MORE INFORMATION GO TO WWW.P6SWARNINGS.CA.GOV.

CAUTIONS:

- Do not use this device if the pouch is damaged or opened.
- Excessive activation may cause handle to overheat.

Applying excessive force on the cautery during use may cause tip to bend or break.

To avoid an undesirable effect, select proper cautery model.

The recommended Duty Cycle is 2 Seconds on and 6 Seconds off for a continuous time of no longer than 15 minutes.

Medical Purpose / Indication
<ul style="list-style-type: none">Bovie® Cautery Devices are used for stopping small bleeders in hemostasis and other similar uses.
Operating Principle
<ul style="list-style-type: none">This is a handheld battery powered device that delivers thermal energy through a heating element to the patient by use of an activation switch.
Patient Population
<ul style="list-style-type: none">No restriction
Site of Use and Site Conditions
<ul style="list-style-type: none">Site of use – No restrictions Site Condition – Aseptic
Intended User Profile
<ul style="list-style-type: none">Education <ul style="list-style-type: none">Trained medical professional Knowledge <ul style="list-style-type: none">Read and understand the supplied IFU Understands hygiene Experience <ul style="list-style-type: none">Some training on techniques or training under surveillance/supervision Permissible Impairments <ul style="list-style-type: none">Mild reading/vision impairment or corrected vision to 20/20

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