## Standard Violet and Blue Skin Markers - Non-Sterile

Aspen Surgical Products, Inc.  
6945 Southbelt Drive SE  
Caledonia, MI 49316 USA  
Phone 616-698-7100  
Toll-Free 888-364-7004  
aspensurgical.com

EMERGO  
EMERGO Europe  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

Made in USA

### English

#### Indications for Use:

Skin marker is used for surgical marking on intact skin

#### Intended User/Patient Target Group:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Precautions:

- The skin marker ink must be applied to intact, dry skin before applying prep solution. The skin marker ink will not adhere to prep solution or wet skin.
- Prepping the skin with isopropyl alcohol before marking may increase the adherence of the ink to the skin.
- When prep solution is applied over the skin marker ink, some of the marking will smear, but the skin marker ink will still be visible.
- This device is single use only. Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.
- Rulers packaged with skin markers are not measuring devices and should be used as an indicator only, and not for accurate measurements.
- Not for ocular use.
- Device is intended to write for up to 24 feet.

#### Sterilization Instructions:

To ensure SAL of 10x-6, this product must be sterilized via gamma sterilization method in its original packaging configuration at a sterilization dose range of 28.0-50.0 Kgy.

#### Instructions for Use:

- Remove the pen cap to expose the pen tip.
- Mark desired areas on skin with the skin marker according to facility protocol.
- Allow the markings to dry on the skin for approximately 1 minute before rubbing or applying prep solution.
- Apply prep solution as instructed by prep solution manufacturer.
- Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Notice to Users and/or Patients in EU:

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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