REF ESMS2

EN - Instructions for Use - Electrosurgical Unit Mobile Cart

INTENDED USE

The Electrosurgical Unit Mobile Cart is a device that is designed as an accessory to allow the easy transportation of the generators and applicable accessories to different places.

This cart is intended to be used with A1250S, A1250U Generators and SE02 Smoke Evacuator only. Do not use with other equipment.

CAUTIONS:

- Periodically inspect functionality of wheel locks, nuts tightness and cart condition.
- Failure to follow assembly instructions may result in equipment damage.
- Failure to follow instructions may result in improperly assembled cart.
- The cart is not sterilized. Do not place sterilized instruments on the cart.
- Clean and disinfect the cart per hospital/facility procedures periodically.
- Do not place other items on the A1250S/A1250U Generator or SE02 Smoke Evacuator. Doing so may reduce the stability of the cart or damage the equipment.
- Verify wheels are locked when cart is stationary.
- Verify position of the Generator is on the corresponding locators on the upper shelf.
- Verify position of the Smoke Evacuator is on the locators on the lower shelf.
- Verify wheels are in the un-lock position when moving cart.
- Verify path is clear of obstacles when moving cart.
- Make sure cart does not tip over when moving cart over threshold.
- **Notice:** For applicable IFU/additional information: Visit www.aspensurgical.com
- **Notice:** After use or upon determining product can no longer be used due to wear or damage, safely discard in accordance with established procedures for biohazardous waste.

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Aspen Surgical Products, Inc.