Symmetry Surgical® (BOOKWALTER®) Universal Table Clamp and Structural Components for BOOKWALTER® Retractor

Symmetry Surgical Inc.
3034 Owen Drive,
Antioch, TN 37013 USA
☎ 1-800-251-3000
Fax: 1-615-964-5566
www.symmetrysurgical.com

[EC REP]

Symmetry Surgical GmbH
Maybachstraße 10
78532 Tuttlingen, Germany

☎ + 49 7461 96490
Fax: + 49 7461 77921

LCN 500-4551-85/S
© 2002–2015 Symmetry Surgical
♻ Revised 06/15

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

Symmetry Surgical® (BOOKWALTER®) Universal Table Clamp and Structural Components for BOOKWALTER® Retractor

General Information

If the instruments are to be used with the BOOKWALTER® Retractor, refer to the instruction manual that provides complete operating instructions and is supplied with the BOOKWALTER® Retractor Kit II (catalog no. 50-4542). Extra copies of this manual are available from Symmetry Surgical®, Antioch, TN 37013 USA. To obtain extra copies outside the United States, contact your local Symmetry Surgical representative. If these instruments are to be used with any other retractor, please refer to that manufacturer's instruction manual.

The structural components for use with this device are the post coupling (catalog no. 50-4554), and either the horizontal bar (catalog no. 50-4552) or the horizontal flex bar (catalog no. 50-4582).

Description

Refer to the package label for a description of contents.

Indications

The BOOKWALTER® Retractor is a self-retaining instrument that provides multi-directional exposure of the abdominal cavity.

WARNINGS

The retractor system can exert a great amount of force on the wound edges and on tissues. Release the retractor for short periods of time or shift the blades to relieve the pressure on any one point.

PRECAUTIONS

If using monopolar electrosurgical equipment while using the BOOKWALTER® Retractor Kit, consult the basic rules of usage concerning electrosurgery (see EN 60 601-2-2), as well as the manufacturer's instructions for use regarding specific warnings and conditions.

Be sure to electrically isolate the patient from (high frequency leakage currents to) the operating table, in accordance with the instructions provided by the manufacturer of the electrosurgical equipment.

Ensure that the patient does not come into contact with metal parts that are earthed or that have an appreciable capacitance to earth (for example: operating table, supports), in accordance with the instructions provided by the manufacturer of the electrosurgical equipment.

Avoid small-area contacts with devices having a low impedance to earth at high frequencies, as unwanted burns may result.

Instructions for Use if Using the Vertical Table Post Or the Symmetry Surgical

BOOKWALTER® Universal Table Clamp Attachment of the Vertical Table Post or Universal Table Clamp

Note: Be sure the operating table side rail does **not** have extra material draped over it. The clamp of both the vertical table post and the Universal table clamp is designed to hold no more than two double thicknesses of cotton drapes.

Pull out the recessed lever, located at the top of the vertical post (see Figures 1 and 2), and fully open the rail clamp by turning the lever in a counter-clockwise direction.

When using the table post, attach the clamp over the drapes as shown in Figure 3. **Note:** If using the Universal table clamp, refer to Figure 4. Retighten the attachment by turning the lever in a clockwise direction as shown in Figure 5.

Attachment of the Post Coupling and Horizontal Bar

If using the table post (catalog no. 50-4581) slide the horizontal bar or horizontal flex bar into either of the holes in the post coupling (50-4554). Slide the post coupling and horizontal bar assembly over the top of the vertical post to its approximate location, and lightly tighten the coupling. The position of the post coupling lever on the vertical post must be convenient to the surgeon. Make any final adjustments before tightening the post coupling (Figure 6).

Note: Follow the same instructions for proper attachment of the post coupling and horizontal bar if using the Universal table clamp.

To position the horizontal flex bar, release the ball joint by turning the lever in a counter-clockwise direction. After positioning the flex bar, tighten by turning the lever in a clockwise direction.

CAUTION: The post coupling bolt should never be tightened unless the post coupling is attached to the bars. The slight deformity of the circular opening that might result could affect the smooth fit between the post coupling and the bars.

Reprocessing Instructions

Refer to Symmetry Surgical Instructions for Use #204233 for updated information regarding reprocessing the instrument.

Copies of this instruction sheet are available by requesting IFU #204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000
Fax: +1 615 964 5566
E-mail: customerservice@symmetrysurgical.com

CAUTION: Only the post coupling can be disassembled for cleaning. Do not attempt to disassemble the table clamp, table post, horizontal bar, horizontal flex bar, or vertical extension bar.

IMPORTANT: The table clamp, vertical table post, horizontal bar, and post coupling should always be lubricated to keep these parts functional.

CAUTION: To ensure proper function do not lubricate the horizontal flex bar (catalog no. 50-4582).

Service and Repair

For service or repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical Inc.
3034 Owen Drive,
Antioch, TN 37013 USA

Always include a repair purchase order number and a written description of the problem. Use only approved authorized dealerships for repairs.

Warranty

Symmetry Surgical® warrants that this medical device is free from defects in both materials and workmanship. Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.

BOOKWALTER® is a registered trademark of Symmetry Surgical Inc.

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Telefone: 800 251 3000

Fax: +1 615 964 5566

Email: customerservice@symmetrysurgical.com

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Symmetry Surgical (BOOKWALTER®)
Universal Table Clamp and
Structural Components for
BOOKWALTER® Retractor

LCN 500-4551-85/S

1

ENGLISH

A. Recessed Lever

FRANÇAIS

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DEUTSCH

A. Eingelassener Hebel

NEDERLANDS

A. Verzonken hendel

2

ITALIANO

A. Leva nascosta

ESPAÑOL

A. Palanca anidada

PORTEGUÊS

A. Alavanca recolhida

Symmetry Surgical (BOOKWALTER®)
Universal Table Clamp and
Structural Components for
BOOKWALTER® Retractor

LCN 500-4551-85/S

3

ENGLISH
A. Rail Clamp

FRANÇAIS
A. Clamp du rail

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NEDERLANDS
A. Stangklem

4

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ESPAÑOL
A. Clamp de riel

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
Symmetry Surgical (BOOKWALTER®)
Universal Table Clamp and
Structural Components for
BOOKWALTER® Retractor

LCN 500-4551-85/S

5

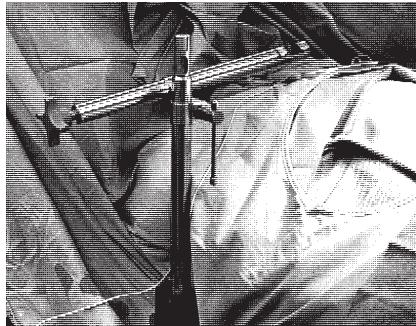6

[EC REP] Authorized European Representative

Représentant agréé pour l'Europe
Autorisierte Vertretung für Europa
Officiële vertegenwoordiging in Europa
Rappresentante autorizzato per l'Europa
Representante autorizado en Europa
Representante autorizado na Europa

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Symmetry Surgical™ (BOOKWALTER®) Universal Table Clamp and Structural Components for BOOKWALTER® Retractor

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