EN AUTOCLAVABLE 3-BUTTON POWER CONTROL HANDPIECE

REF A902 – For use with A952 ESU and DERM 101/102/941/942 desiccators

- Store At Room Temperature
- Collect 092° Min. - 024° Max
- Device is rated for 6300 Vpeak

WARNING: Do not use in the presence of flammable materials. Fire could result.

WARNING: Improper electrode installation may result in injury to the patient or operating room personnel by arcing at the electrode pencil connection.

WARNING: To avoid burn or shock injury, turn off the desiccator at the mains power. **WARNING:** To avoid burn or shock injury, do not activate pencil while wiping electrode tip during use.

WARNING: Keep active accessories away from the patient when not in use. Accessories and electrodes should be placed in a clean, dry, nonconductive and highly visible area when not in use.

WARNING: Inspect prior to use. The device should be replaced if any physical damage such as nicks, cracks or cuts is noted on the handpiece or cable.

CAUTION: This product is NON-STERILE. The use of a sterile sheath or sterilizing prior to use is recommended.

CAUTION: The device is designed for use only with the A952 ESU and DERM 101/102/941/942 desiccators that will deliver an optimal surgical effect.

DIRECTIONS FOR USE:

- Place electrode into the handpiece. Ensure the electrode is fully inserted.
- Plug the handpiece into the monopolar output jacks on the lower left of the front of the unit. The plug is designed to fit in only one direction.
- To remotely adjust the power settings on the unit press the up or down button on the handpiece. Depressing and holding the up or down buttons will cause the power settings to change more rapidly.
- To activate the monopolar output, depress the "ON" button on the handpiece. This is the button located nearest the electrode.
- After use, remove electrode and resterilize the pencil following the recommended procedure as follows.

CLEANING INSTRUCTIONS:

- Remove all gross matter (blood, tissue) by wiping down the pencil using a gauze pad or soft cloth dampened in warm tap water until all visible debris is removed.
- Wipe the pencil using a gauze pad or soft cloth dampened in an enzymatic detergent mixture diluted according to the manufacturer’s instructions. The pencil may also be gen-ly scrubbed with a soft cleaning brush.
- IMPORTANT:** Abrasive material should not be used in the area of the barcode when cleaning the device.
- Remove cleaning agents from pencil by wiping with tap water dampened cloth.

STERILIZATION INSTRUCTIONS:

Number of uses when sterilized by steam autoclave: 25

- Steam autoclave at 250° F (121° C, gravity) for 20 minutes using an autoclavable pouch or wrapped cloth method. Consult the AAMI recommended practice for steam steriliza- tion using the strapped cloth method. Allow product to cool at least 30 minutes before use.
- Flash autoclave method is not recommended.

Medical Purpose/ Indication
The Device is an accessory to the DERM101/DERM102 to serve as the pathway to deliver RF energy via an assortment of RF- electrodes for heat coagulation of different kinds of tissues.

Operating Principle
The device is used in conjunction with an electro-surgical generator and replaceable electrodes for the delivery of radiofrequency (RF) current. The device is to be used with a variety of monopolar electrodes for the coagulation and desiccation of tissue.

Patient Population
No restrictions

Site of Use and Site Conditions
Site of use – Soft tissue
Site conditions – Aseptic

Intended User Profile
This device is intended to be used by trained physicians or individuals familiar with electro-surgery.

Intended Conditions for Use
Education

- Trained physician, physicians assistant, nurse or nurse practitioner

Knowledge

- Understands electro-surgery and electro-surgical techniques
- Has read and understands supplied Instructions for Use or Device Labeling

Experience

- Some training on techniques or training under surveillance/supervision

Permissible impairments

- Mild reading vision impairment or corrected vision to 20/20

Intended Conditions for Use

- Doctor office, Surgery center or Hospital, intended for professional use only
- Clear and unobstructed view at point of use
- Ambient Temperature 10° C to 40° C

Frequency of use

- The A902 Pencil is reusable and designed to withstand a maximum of 25 gravity autoclave sterilization cycles

Disposal

- Biohazard – Dispose of Pencil according to facility procedures

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