## U-Bag® Urine Collectors

Instructions for Use

<sup>[US]</sup> **Applying the Single Specimen U-Bag®** Separate the legs of the infant or child; make sure perineal area is clean and dry. Do not apply powders, oils or lotions to skin.

- Remove the release paper from the lower half of the adhesive patch.
- Stretch perineum to remove skin folds. Place the narrowest section of the exposed adhesive against the perineum. In female patients the perineum is the arrow bridge of skin between the vagina and the anus.
- In males the perineum is the skin between the scrotum and the anus. Press the adhesive patch firmly against the skin to ensure proper adhesion.

Working outward from this point, smooth the rest of the exposed adhesive against the skin. Avoid wrinkles which can create channels for leakage.

Remove the release paper from the upper portion of the adhesive patch once the lower portion of the adhesive is securely in place. Working upward and outward, smooth the adhesive against the skin, pressing the adhesive firmly to avoid any wrinkles.

**Removing and Draining the U-Bag®**

To remove the collector, start at the edge and gently peel the adhesive patch from the skin.

- Tilt the bag so urine is away from the blue tab. Remove tab and drain into clean receptacle.

**Note:** Since the Premie U-Bag® has no drain tab, fold the adhesive patch in half so it adheres to itself and send the U-Bag to the lab.

**Caution:** As with any adhesive device, the adhesive component of these products may cause mild irritation upon removal.

**Applying the 24-Hour U-Bag®**

Remove the collector, the coiled drain tube, and the junction and cap from the card.

Apply the 24-hour collector in the same manner described for the single specimen collector.

Direct the drainage tube into a receptacle.

**Adjusting Drain Tube Length**

To allow the child freedom of movement, the tube can be temporarily shortened and capped, thus permitting a limited quantity of urine to collect in the bag and attached tubing.

- Cut the tube anywhere along its length, preferably 8 to 13 cm below the bag.
- Insert the plastic junction and place the black plastic cap on the end (cap and junction are enclosed).
  - To drain the filled bag, lift the shortened tubing, remove the cap, and empty the urine into a receptacle.
- To resume collection, remove the cap (to lengthen drain, gently attach extra tubing to the junction) and direct the end into a bedside collector.

**WARNING:** To help reduce the potential for infection and/or other complications, do not re-use.

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