INSTRUCTIONS FOR USE

Bard-Parker®

Reusable Handles

Non-Sterile

aspensurgical.

Aspen Surgical Products Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316
Phone +1 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

For Country of Origin See Product Label

English

Reusable Handles

Intended Use

Intended to be used with surgical blades for tissue separation and other procedures that require a sharp surgical blade to puncture or cut.

Intended User/Patient Target Groups

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instructions for Use:

- Ruler shown on scalpel handle should be used as an indicator only, and not for accurate measurements.
- During use avoid twisting, bending or putting excessive force or strain on the blade in order to help prevent blade breakage.
- A singular blade is intended to cut tissue up to 24 inches.
- Do not use if product appears visually damaged or degraded.

Cleaning Procedure – Hand Washing

- After use, carefully remove scalpel blade and discard it. Do not allow blood or debris to dry on the handle.
- Immerse the scalpel handle in an enzymatic detergent prepared according to manufacturer's instructions and soak for 10 minutes.
- Scrub handle with a soft bristled brush (agitate while scrubbing) paying special attention to the surface area where the blade was attached.
- Rinse with warm tap water (38-49°C) for 1 minute.
- Wipe with a clean, lint free cloth to dry device.
- When dry, inspect device under magnification for any visible debris. If debris is visualized on the device surface after cleaning, repeat cleaning to remove all soils.

Cleaning Procedure – Washer/Disinfecter

- After use, carefully remove scalpel blade and discard it. Do not allow blood or debris to dry on the handle.
- Rinse the handle in cool water to remove any visible soil.
- Place items in the basket of the washer/disinfecter, handle side down.
- Run a typical cycle including a dry phase. Cycle used consisted of:
 - Prewash, 2' cold water
 - Enzyme wash, 4' hot tap water with Enzol (1 oz/gal)
 - Wash 1, 2' 65.6° with Valsure (0.25 oz/gal)
 - Rinse 1, 0:15 with hot tap water
 - Dry, 6' 98.8°
- Remove handles and inspect them under magnification for any visible debris.

Sterilization Procedure

Place handles directly into a peel pouch. Do not wrap the handles.

Using a Pre-Vac Autoclave, sterilize using the following parameters:

Cycle Parameter	Specification
Cycle Type	Pre-Vac
Sterilization Temperature	≥132° C
Sterilization Time	4 minutes
Dry Time	20 minutes

Disposal

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional and/or local laws and regulations.

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