

## Instructions for use

### Dr. Don's® Arm Canoe - V 711501

Previously Beatty® reorder # 020801-V

#### Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

#### Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Precautions:

- For use with the ARTHREX® Lateral Decubitus Shoulder Traction Tower, ARTHREX® 3-Point Shoulder Distraction System.
- When hooking the canoe to the positioning device ensure weight is maintained on the device to avoid the arm from dropping.
- Ensure wrap is properly secured to the arm to avoid possible slippage and contamination of the sterile field.
- This device is single use only.
- Do not re-sterilize.
- Do not use if product is damaged or particulate is found in sterile packaging.

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Instructions for Use:

1. Place arm in foam holder after it has been aseptically prepped for surgery.
2. Wrap with self-adhering wrap tightly.
3. Hook to the distraction device and apply the manufacture's recommended weight to the device for optimum shoulder distraction.
4. Discard after use.

Arthrex® is a registered trademark of Arthrex, Inc. ("Arthrex"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Arthrex Inc.