Boot Liner

Non-Sterile

Lindy Traction Boot Liner 712900

Patient Care Boot Liner 712300

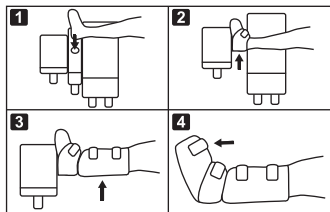
Aspen Surgical Products, Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316 USA
Phone +1 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

EC REP
Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

For Country of Origin See Product Label

English

Intended Use

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User/Patient Target Groups

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instruction for Use:

- Place liner with facing down.
- Place patient's foot in liner, aligning the patient's heel with round opening.
- Follow illustrated directions for securing foam around patient's foot.
- Ensure boot straps are loose.
- Place patient's wrapped foot into boot and secure using manufacturer's recommendations.
- Secure boot to table using manufacturer's recommendations.
- Device is intended for limited contact duration, less than or equal to 8 hours, less if damaged.
- Discard after use.

Precautions:

- 712900: For use with Allen™ Traction Boot A – 93000 and Steris® OT 1000 Traction Boot.
- 712300: Allen™ Traction Boot A – 93005 and A – 93000 and Steris® Traction Boot.
- A – 93000 and Steris® Traction Boot.
- Ensure liner is securely fastened around foot before placing in the positioning device.
- Avoid redundant material that may wrinkle or cause excess pressure.
- This device is single use only.
- Do not sterilize.

Warning

Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.

Disposal

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional and/or local laws and regulations.

Notice to Users and/or Patients in EU

Any serious incident that has occurred in relation to the device should be reported to Aspen Surgical Inc., the governing body, and the Competent authority of the member state in which the incident occurred.

Allen™ is a registered trademark of Allen Medical Systems, Inc. ("Allen Medical"). Steris® is a registered trademark of Steris Inc. ("Steris"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Allen Medical or Steris.

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Emergo Consulting (UK) Limited