INSTRUCTIONS FOR USE

Bard Parker®

aspensurgical.

Protected Disposable Scalpel

Important Usage Guidelines

Aspen Surgical Products, Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316 USA
Phone +1 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

EC REP

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands
For Country of Origin See Product Label
8365993 | DGP69 | REV 1 | 10CT2023

CE
2797

English

Bard-Parker® Protected Disposable Scalpel
Important Usage Guidelines

Intended Use

Intended for tissue separation and other procedures that require a sharp blade to puncture or cut.

Intended User/ Patient Target Groups

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instructions for Use:

- Ruler shown on scalpel handle should be used as an indicator only, and not for accurate measurements.
- During use avoid twisting, bending or putting excessive force or strain on the blade in order to help prevent breakage.
- A singular blade is intended to cut tissue up to 24 inches.
- If blade becomes dull or breaks, dispose and replace product.
- If safety shield or housing malfunction, dispose and replace product.
- Do not use product if product is damaged.

To Use:

- Depress top button on protective shield.
- Retract shield to rear locked position to fully uncover blade

To Cover:

- Depress top button on protective shield.
- Advance shield to forward locked position at lock indicator line.

Warning

Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.

Disposal

Dispose in approved puncture-resistant sharps container according to facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Notice to Users and/or Patients in EU

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Emergo Consulting (UK) Limited