
## INSTRUCTIONS FOR USE

aspen surgical.

### Sterile Probe Cover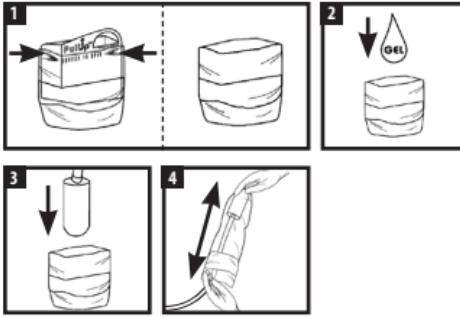

### English

#### Intended Use

These protective covers are medical devices used over imaging probes or instruments to provide a sterile field and minimize cross contamination during diagnostic imaging or surgical procedures, operating on the body surface, intracavity, and intraoperative procedures.

Protective covers help prevent transfer of microorganisms, body fluids, and particulate matter to the patient and healthcare provider during procedures using ultrasound transducers/imaging instruments and systems.

#### Intended User/Patient Target Groups

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Instructions for Use

Prepare probe per system's operational manual.

1. Open probe cover (styles vary).
2. Place gel inside cover and/or on probe scanning surface.
3. Insert probe into cover.
4. Pull cover over probe and cord. Use tape or bands to fasten cover. Use sterile gel between cover and patient.
5. The device is intended for continuous use for less than 60 minutes.
6. Do not use if package is damaged.

#### Warning

Single Use Devices Only: Reusing or reprocessing single use devices may cause damage to the device which in turn may cause unnecessary harm to the user and/or patient. Reuse or reprocessing is not recommended.

#### Disposal

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Notice: After Use or upon determining product can no longer be used due to wear or damage, safely discard in accordance with established procedures for biohazardous waste.

NOTICE: Any serious incident that has occurred in relation to the device should be reported to Aspen Surgical Products, Inc., the governing body, and the Competent authority of the member state in which the incident occurred.

Notice: For applicable IFU/additional information: Visit [aspensurgical.com](http://aspensurgical.com)

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Emergo Consulting (UK) Limited
Aspen Surgical Products Inc.