SURGICAL LIGHT (Sterile)

EN

REF

ST05, ST10, ST15

DESCRIPTION:
The Surgical Light is a sterile, battery operated, single-use device available in 5 inch, 10 inch, and 15 inch flexible shafts. The light is used for general illumination inside body cavities during surgery.

INDICATIONS FOR USE:
The device is indicated for use where additional illumination is needed within the sterile field.

CONTRAINDICATIONS:
The device is contraindicated for use in the presence of flammable gases or other materials capable of ignition.

• STORE AT ROOM TEMPERATURE • LIGHT HAS A TYPICAL BURN LIFE OF APPROXIMATELY 2 HOURS

WARNINGS:
Do not use in the presence of flammable gases/materials or oxygen-enriched environments. Fire could result.

This medical device cannot be effectively cleaned and/or sterilized by the user and therefore cannot be safely reused. It is intended for single use only. Any attempt by the user to clean and resterilize this device may result in bioincompatibility, infection, or other risks of device failure to the patient.

 **WARNING:** This product is sterilized utilizing Ethylene Oxide. These products can expose you to chemicals including Ethylene Oxide (EtO), which is known to the state of California to cause cancer and/or birth defects or other reproductive harm. For more information go to **www.P65Warnings.ca.gov**.

CAUTIONS:

Do not cut the flexible shaft.

Avoid direct contact with tissue.

Avoid having bulb make contact with hard objects.

IMPORTANT: After use, use proper disposal method.

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