**Symmetry Surgical™
BOOKWALTER®
Post Holder and
Symmetry Surgical™
BOOKWALTER®
Goose Neck Table Post**

Symmetry Surgical
3034 Owen Drive
Antioch, TN 37013 USA
Phone: 1-800-251-3000
Fax: 1-615-964-5566
www.symmetrysurgical.com

Symmetry Medical Sheffield
Beulah Road
Sheffield, England
S6 2AN
☎ +44-114-285-5881
Fax: +44-114-233-6978

LCN 205183-001/D

© 2006–2012 Symmetry Surgical
♻️ Revised 07/13

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

Symmetry Surgical™ BOOKWALTER® Post Holder and Symmetry Surgical™ BOOKWALTER® Goose Neck Table Post

Rx Only

Description

If the instruments are to be used with the BOOKWALTER Retractor, refer to the instruction manual that provides complete operating instructions and is supplied with the BOOKWALTER Retractor Kit II (catalog no. 50-4542). Extra copies of this manual are available from Symmetry Surgical, 3034 Owen Drive, Antioch, TN 37013 USA. To obtain extra copies outside the United States, contact your local Symmetry Surgical representative. If these instruments are to be used with any other retractor, please refer to that manufacturer's instruction manual.

Refer to the package label for a description of contents.

Indications

The BOOKWALTER Retractor is a self-retaining instrument that provides multi-directional exposure of the abdominal cavity.

WARNINGS

The retractor system can exert a great amount of force on the wound edges and on tissues. Release the retractor for short periods of time or shift the blades to relieve the pressure on any one point.

PRECAUTIONS

If using monopolar electrosurgical equipment while using the BOOKWALTER Retractor Kit, consult the basic rules of usage concerning electrosurgery (see EN 60 601-2-2), as well as the manufacturer's instructions for use regarding specific warnings and conditions.

Be sure to electrically isolate the patient from (high frequency leakage currents to) the operating table, in accordance with the instructions provided by the manufacturer of the electrosurgical equipment.

Ensure that the patient does not come into contact with metal parts that are earthed or that have an appreciable capacitance to earth (for example: operating table, supports), in accordance with the instructions provided by the manufacturer of the electrosurgical equipment.

Avoid small-area contacts with devices having a low impedance to earth at high frequencies, as unwanted burns may result.

Instructions for Use

Attachment of the Symmetry Surgical BOOKWALTER Post Holder

Note: Be sure the operating table side rail does **not** have extra material draped over it. The clamp of the vertical table post is designed to hold no more than two double thicknesses of cotton drapes.

Lift the tab located on top of the post holder and turn in a counter-clockwise direction to fully open the post holder clamp (Figure 1).

Attach the post holder over the drapes as shown in Figure 2 and retighten the post holder by turning the lever in a clockwise direction.

Attachment of the Symmetry Surgical BOOKWALTER Goose Neck Table Post

Slide the BOOKWALTER Goose Neck table post (catalog no. 50-4681) into the post holder's integral post coupling and secure the attachment by turning the lever in a clockwise direction. The position of the post coupling lever on the table post should be convenient to the surgeon. Make any final positioning adjustments before tightening the post coupling.

To position the horizontal bar (catalog no. 50-4552) or horizontal flex bar (catalog no. 50-4582), release the ball joint by turning the lever in a counter-clockwise direction.

After positioning the horizontal bar or horizontal flex bar, tighten by turning the lever in a clockwise direction. See Figure 3 for proper assembly.

CAUTION: The post coupling bolt should never be tightened unless the post coupling is attached to the bars. The slight deformity of the circular opening that might result could affect the smooth fit between the post coupling and the bars.

Reprocessing Instructions

Refer to Symmetry Surgical Instructions for Use #204233 for updated information regarding reprocessing the instrument.

Copies of this instruction sheet are available by requesting IFU #204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000

Fax: +1 615 964 5566

E-mail:

ssi-customerservice@symmetrysurgical.com

CAUTION: Do not attempt to disassemble the post holder, horizontal bar, horizontal flex bar, or vertical extension bar before cleaning.

IMPORTANT: The post holder and horizontal bar must always be lubricated to keep these parts functional.

CAUTION: To ensure proper function do not lubricate the horizontal flex bar (catalog no. 50-4582).

Service and Repair

For service or repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical
3034 Owen Drive,
Antioch, TN 37013 USA

Always include a repair purchase order number and a written description of the problem. Use only approved authorized dealerships for repairs.

Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

® BOOKWALTER is a registered trademark of Symmetry Surgical

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