Leg Positioning

Non-Sterile

Parent Post Knee Pad 712500

Thigh Holder Pad 713500

Leg Comfort Pad 713600

Aspen Surgical Products, Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316 USA
Phone +1 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

EC REP
Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

For Country of Origin See Product Label

English

Intended Use

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User/Patient Target Groups

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instruction for Use

- For 712500: Slide over post.
- For 713500: Place thigh in holder on the pad.
- For 713600: Place pad on upper thigh and place leg in positioning device ensuring it is secure.
- Device is intended for limited contact duration, less than or equal to 8 hours, less if damaged.
- Discard after use.

Compatibility Chart:	
ARTHREX® PIVOTPOST® AR-1560.	Parent Post Knee Pad 712500
Smith & Nephew® FERKEL™ Thigh System 72201812.	Thigh Holder Pad 713500
Allen® Leg Holder R-734-33, Smith & Nephew® Acufex Knee Holder, SchureMed™ Schure Grip Leg Holder 800-0065, and Stryker® Arthroscopy Leg Holder OZ 7511975.	Leg Comfort Pad 713600

Precautions:

- Ensure that the pad is completely secure to patient and/or device.
- For 713500: Ensure thigh is completely in holder after placing in the positioner.
- Be aware of skin to avoid pinching while placing patients' leg in positioning device.
- For 713600: Do not place over a tourniquet.
- This device is single use only.
- Do not sterilize.

Warning

Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.

Disposal

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional and/or local laws and regulations.

Notice to Users and/or Patients in EU

Any serious incident that has occurred in relation to the device should be reported to Aspen Surgical Inc., the governing body, and the Competent authority of the member state in which the incident occurred.

Arthrex® and PivotPost® AR-1560 are registered trademarks of Arthrex, Inc. ("Arthrex"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Arthrex.

Allen® is a registered trademark of Allen Medical Systems, Inc. ("Allen Medical").

Smith & Nephew® is a registered trademark of Smith & Nephew PLC (Smith & Nephew). SchureMed™ is a registered trademark of Schuerch Corporation ("Schuerch"). Stryker® is a trademark of Stryker Corporation ("Stryker"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Allen Medical, Smith & Nephew, SchuerchMed, or Stryker.

Smith & Nephew® and FERKEL™ are registered trademarks or trademarks of Smith & Nephew PLC (Smith & Nephew). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Smith & Nephew.

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Emergo Consulting (UK) Limited