
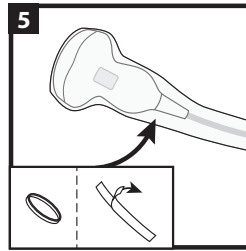
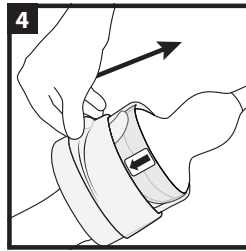
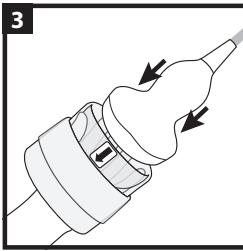
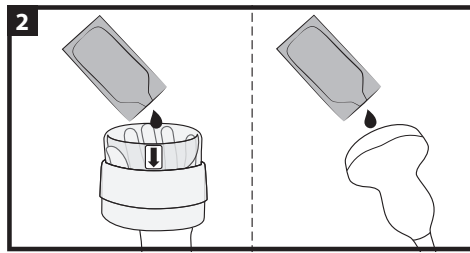
EN / English	Instructions for Use Sterile Single-Use Protective Covers
DE / Deutsch	Gebrauchsanweisungen Sterile Einweg-Schutzhüllen
FR / Français	Mode d'emploi Protections stériles à usage unique
BG / Български	Инструкции за употреба Стерилни защитни обвивки за еднократна употреба
CS / Čeština	Návod k použití Jednorázové sterilní ochranné kryty
DA / Dansk	Brugsanvisning Sterile beskyttelsesdækker til engangsbrug
EL / Ελληνικά	Οδηγίες χρήσης Αποστειρωμένα προστατευτικά καλύμματα μιας χρήσης
ES / Español	Instrucciones de uso Cubiertas protectoras estériles desechables
ET / Eesti Keel	Kasutusjuhend Steriilsed ühekordselt kasutatavad kaitsekatted
FI / Suomi	Käyttöohje Ei-steriilit kertakäyttöiset suojuukset
HR / Hrvatski	Upute za upotrebu Sterilne zaštitne navlake za jednokratnu upotrebu
HU / Magyar	Használati utasítás Steril, egyszer használatos védőburkolat
IT / Italiano	Istruzioni per l'uso Coperture protettive sterili monouso
KO / 한국어	사용 설명서 멸균 일회용 보호 커버
LT / Lietuvių k.	Naudojimo instrukcijos Sterilus vienkartinio naudojimo apsauginiai gaubtai
LV / Latviešu	Lietošanas instrukcija Sterili vienreizlietojami aizsargpārvalki
NL / Nederland	Gebruiksaanwijzing Niet-steriele beschermhoezen voor eenmalig gebruik
NO / Norsk	Bruksanvisning Sterile beskyttelsesdeksler til engangsbruk
PL / Polski	Instrukcja użycia Jałowe jednorazowe osłony ochronne
PT / Português	Instruções de utilização Capas protetoras descartáveis esterilizadas
RO / Română	Instrucțiuni de utilizare Huse de protecție de unică folosință sterile
RU / Русский	Инструкции по применению Стерильные одноразовые защитные чехлы
SK / Slovensky	Návod na na použitie Sterilné ochranné kryty na jedno použitie
SL / Slovenščina	Navodila za uporabo Sterilni zaščitni pokrovi za enkratno uporabo
SR / Srpski	Uputstvo za upotrebu Sterilni zaštitni pokrivači za jednokratnu upotrebu
SV / Svenska	Bruksanvisning Sterila skyddshöljen för engångsbruk
TR / Türkçe	Kullanım İçin Talimatlar Steril Tek Kullanımlık Koruyucu Kılıflar
ZH / 中国人	使用说明 无菌一次性防护套
JA / 日本語	使用手順 滅菌シングルユース保護カバー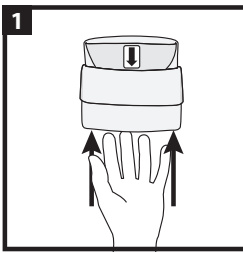

EN / English

Prepare probe per system's operational manual. After procedure, to remove the probe cover, follow the steps shown for mounting on probe, in reverse order.

INTENDED PURPOSE/INTENDED USE | These protective covers are medical devices used over imaging probes or instruments to provide a sterile field and minimize cross contamination during diagnostic imaging or surgical procedures, operating on the body surface, intra-cavity and intra-operative procedures. Protective covers help to prevent transfer of microorganisms, body fluids, and particulate matter to the patient and healthcare provider during procedures using ultrasound transducers/imaging instruments and systems.

INTENDED USER | These devices are used by healthcare professionals trained and qualified on using these types of devices.

OTHER DEVICES INTENDED TO BE USED | Protective covers are used with ultrasound probes or imaging instruments and ultrasound transmission gel (water base).

BEFORE PERFORMING PROCEDURE | Verify proper fit on the probe/instrument/system. Cover is to be placed tightly over transducer face to eliminate wrinkles and air bubbles, taking care to avoid piercing cover.

WARNING

- This product is single-use only. Do not reuse. Do not re-sterilize. Reuse or reprocessing of the single use devices may create a risk of cross contamination, patient infection, and/or malfunction of the device.
- Contents sterile if package is intact. If sterile packaging is damaged or unintentionally opened discard and use an intact package.
- Do not use if found damaged or does not fit properly. Discard and use an intact package.
- Malfunction: In the case of any signs of a malfunction or changes in performance which may affect safety (e.g., holes or cuts in cover), stop procedure, discard defective cover, use applicable contamination control methods to prevent patient infection, and replace cover with a new one.
- Storage conditions: Keep away from direct sunlight or extreme temperatures/heat sources. Store in a cool, dry place.
- Disposal: After use, dispose of single-use components as infectious waste.

Notice: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user is established.

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Aspen Surgical Products, Inc.