INSTRUCTIONS FOR USE

Bard Parker®

aspen surgical.

Protected Blade System

STERILE R R_X ONLY MD

EC REP

Aspen Surgical Products, Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316 USA
Phone +1 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

For Country of Origin See Product Label

8365994 | DGP70 | REV10 | 11JAN2024

CE
2797

English

Bard-Parker® Protected Blade System

Intended Use

Intended for tissue separation and other procedures that require a sharp blade to puncture or cut.

Intended User/ Patient Target Groups

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instructions for Use:

- Ruler shown on scalpel handle should be used as an indicator only, and not for accurate measurements.
- During use avoid twisting, bending or putting excessive force or strain on the blade in order to help prevent breakage.
- If blade becomes dull or breaks, dispose and replace product.
- If safety shield or housing malfunction, dispose and replace product.
- A singular blade is intended to cut tissue up to 24 inches.
- Do not use product if product is damaged.

Assemble:

- Grasp protective shield with thumb and index finger with blade tip pointing up.
- Insert handle cleat and attach protected blade up to the indicator line.

Caution: DO NOT place hand in front of shield opening

Use:

- Depress top button on protective shield.
- Retract shield to rear locked position to fully uncover blade.

Cover:

- Depress top button on protective shield.
- Advance shield to forward locked position at lock indicator line.

Dispose-One-handed technique with blade covered:

- Place thumb on side of handle behind cartridge.
- Push forward to unload protected blade.

Disposal

Dispose in approved puncture-resistant sharps container according to facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Warning

Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.

Notice to Users and/or Patients in EU

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Emergo Consulting (UK) Limited