INSTRUCTIONS FOR USE

Bard Parker®

aspensurgical.

Conventional Disposable Scalpel

STERILE

Rx ONLY

MD

EC REP

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands
Made in USA

CE
2797

Aspen Surgical Products, Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316 USA
Phone 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

8365942 | DGP61 | REV1 | 01DEC2020

English

Bard-Parker® Conventional Disposable Scalpel

Intended Use:

Intended for tissue separation and other procedures that require a sharp surgical blade to puncture or cut.

Intended User/ Patient Target Groups:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- This device is single use only. Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.
- Ruler shown on scalpel handle should be used as an indicator only, and not for accurate measurements.
- During use avoid twisting, bending or putting excessive force or strain on the blade in order to help prevent breakage.
- A singular blade is intended to cut tissue up to 24 inches.
- If blade becomes dull or breaks, dispose and replace product.
- Do not use product if product is damaged or particulate is found in sterile packaging

Disposal:

Dispose in approved puncture-resistant sharps container according to facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Notice to Users and/or Patients in EU:

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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