The BOOKWALTER® IMA Retractor Kit

Symmetry Surgical Inc.
3034 Owen Drive,
Antioch, TN 37013 USA
☎ 1-800-251-3000
☎ 1-615-964-5566
www.symmetrysurgical.com

EC REP

Symmetry Surgical GmbH
Maybachstraße 10
78532 Tuttlingen, Germany
☎ +49 7461 96490
☎ +49 7461 77921

CE 2797

These instructions pertain to both
Class I and Class II devices.

LCN 170088-001/P
© 2002–2020 Symmetry Surgical
♻ Revised 08/2020

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

The BOOKWALTER® IMA Retractor Kit

Indications

The BOOKWALTER® IMA Retractor Kit (catalog no. 50-4600) provides the tools to elevate and retract the sternum for efficient dissection of the internal mammary artery.

Description

The kit is comprised of the following components, which also may be purchased separately (Figure 1).

Component	Quantity	Catalog No.
A. Horizontal Bar	1	50-4552
B. Post Coupling	1	50-4554
C. Table Post	1	50-4581
D. IMA Cross Bar	1	50-4601
E. IMA Ratchet Mechanism	2	50-4602
F. IMA Retractor Blade, 12 inch	2	50-4603
G. IMA Storage Case	1	50-4604

Note: The Symmetry Surgical BOOKWALTER® IMA T-Cross Bar (catalog no. 50-4675) is available separately and can be used in place of the IMA Cross Bar (Figure 2). Please refer to “Attachment of the T-Cross Bar and Retractor Blades” in this manual for proper attachment.

Assembly of the system is simple, and the components are self-contained to make them easily accountable.

If the instruments are to be used with the BOOKWALTER® Retractor, refer to the instruction manual that provides complete operating instructions and is supplied with the BOOKWALTER® Retractor Kit II (catalog no. 50-4542). Extra copies of this manual are available from Symmetry Surgical, Antioch, TN 37013 USA. To obtain extra copies outside the United States, contact your local Symmetry Surgical representative. If these instruments are to be used with any other retractor, please refer to that manufacturer’s instruction manual.

WARNINGS

The retractor system can exert a great amount of force on the wound edges and on tissues. Release the retractor for short periods of time or shift the blades to relieve the pressure on any one point.

PRECAUTIONS

Do not tighten the post coupling bolt unless the coupling is attached to the horizontal bar. The slight deformity of the circular opening that might result could affect the smooth fit between the post coupling and the bars.

If using monopolar electrosurgical equipment while using the BOOKWALTER® IMA Retractor Kit, consult the basic rules of usage concerning electrosurgery (see EN 60601-2-2), as well as the manufacturer’s instructions for use regarding specific warnings and conditions.

Be sure to electrically isolate the patient from (high frequency leakage currents to) the operating table, in accordance with the instructions provided by the manufacturer of the electrosurgical equipment.

Ensure that the patient does not come into contact with metal parts that are earthed or that have an appreciable capacitance to earth (for example: operating table, supports), in accordance with the instructions provided by the manufacturer of the electrosurgical equipment.

Avoid small-area contacts with devices having a low impedance to earth at high frequencies, as unwanted burns may result.

Assembly

Attaching the Table Post

1. Pull out the recessed lever, located at the top of the vertical post (Figures 3 and 4). Turn the lever counterclockwise to fully open the rail clamp.

Note: Make sure the operating table side rail does not have extra material draped over it. The clamp of the table post should not hold more than two double thicknesses of cotton drape.

2. Attach the clamp over the drapes (Figure 5).

3. Tighten the clamp to secure the post by turning the lever clockwise.

Attaching the Post Coupling and Horizontal Bar

1. Slide the horizontal bar into either of the holes in the post coupling.

2. Slide the post coupling and horizontal bar assembly over the top of the vertical post, to the desired position, and tighten the coupling

by turning its knob clockwise. Make sure the bar's knob is on the side of the table post that is away from the patient.

CAUTION: Do not tighten the post coupling bolt unless the coupling is attached to the horizontal bar. The slight deformity of the circular opening that might result could affect the smooth fit between the post coupling and the bars.

Attaching the Cross Bar and Retractor Blades

1. Place the flattened portion of the cross bar into the horizontal bar's niche (Figure 6). The cross bar should be parallel with the patient's chest.
2. Make sure the clasps on the ends of the cross bar are within the bar (Figure 7).
3. Slide the IMA Retractor blades into the ratchet mechanisms. Then, slide the ratchet mechanisms onto the cross bar.
4. To lock the ratchet mechanisms onto the bar, turn the clasps on the ends of the cross bar perpendicular to the bar.
5. To effect retraction of either retractor blade, turn the thumb piece on the ratchet mechanism clockwise.

Note: To remove a retractor blade from a ratchet mechanism, press the button on the ratchet mechanism and slide the retractor blade out of the ratchet mechanism.

Attaching the T-Cross Bar and Retractor Blades

1. Slide the IMA Retractor blades into the ratchet mechanisms. Next, slide the ratchet mechanisms onto the T-cross bar.
2. To lock the ratchet mechanisms onto the T-cross bar turn the thumb piece on the ratchet mechanism clockwise and tighten to desired position.

See **Note** above for removing retractor blades from the IMA T-Cross Bar.

Reprocessing Instructions

Refer to Symmetry Surgical Instructions for Use #204233 for updated information regarding reprocessing of the instrument.

Copies of this instruction sheet are available by requesting IFU #204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000
Fax: +1 615 964 5566
E-mail:
ssi-customerservice@symmetrysurgical.com

Service and Repair

For service or repair outside the United States, contact your local Symmetry Surgical representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical
3034 Owen Drive,
Antioch, TN 37013 USA

Always include a repair purchase order number and a written description of the problem. Use only approved authorized dealerships for repairs.

Warranty

Symmetry Surgical warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

BOOKWALTER® is a registered trademark of Symmetry Surgical

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