INSTRUCTIONS FOR USE

Richard-Allan®

Surgical Needles

aspen surgical.

STERILE

Rx ONLY

MD

EC REP

Aspen Surgical Products, Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316 USA
Phone +1 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

For Country of Origin See Product Label

CE
2797

English

Intended Use

Surgical Needles are Intended to be used with suture material for tissue closure.

Intended User/ Patient Target Groups

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instructions for Use:

- Improper surgical technique in the use of a suturing needle may result in needle breakage.
- Improper needle selection for the surgery intended may also result in needle breakage. Needle selection is the responsibility of the surgeon.
- Small needle holders should be used on fine gauge needles and heavy holders on heavy gauge needles to avoid needle breakage because of undue stress.
- To avoid breakage, the needle holder (for example, the jaws of the forceps) should be in good condition and show no signs of deterioration and should be applied to the flat portion of the needle about 1/4 of a needle length from the eyed end.
- A singular needle containing suture material is intended to puncture through tissue up to 30 times.
- Do not use product if product is damaged.

Warning

Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.

Disposal

Dispose in approved puncture-resistant sharps container according to facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Notice to Users and/or Patients in EU

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Emergo Consulting (UK) Limited