
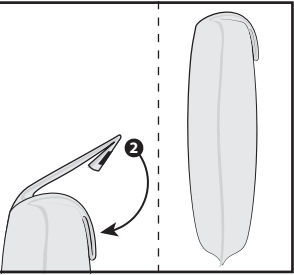
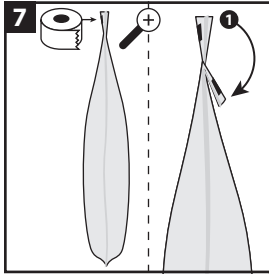
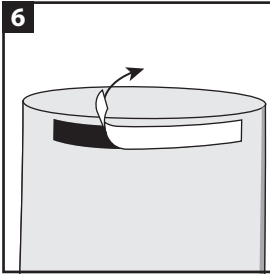
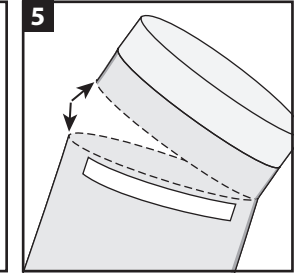
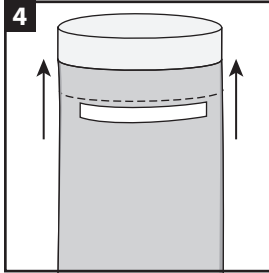
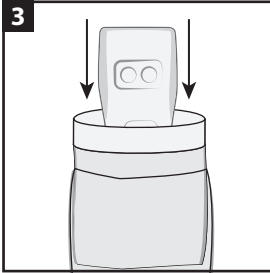
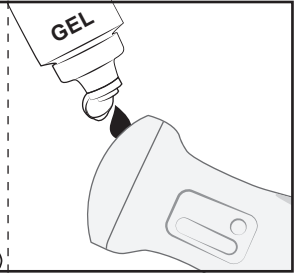
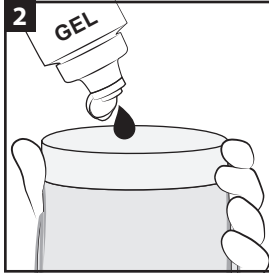


# Instructions For Use

aspensurgical.

## Protek™ PullUp™ Probe Cover

ULTRASOUND ACCESSORIES

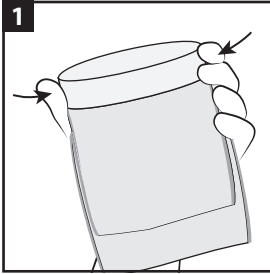

### PREPARE PROBE PER SYSTEM'S OPERATIONAL MANUAL

- 1 Squeeze sides to open cover.
- 2 Place gel inside cover and/or on probe scanning surface.
- 3 Insert probe into cover.
- 4 Pull cover over probe.
- 5 Detach PullUp™ insert along perforation.

- 6 Remove tape liner.
- 7 Fold end of cover (2x) and close with tape. Use bands to fasten cover.
- 8 Use sterile gel between cover and patient.
- 9 After use, this product may be a potential biohazard. Handle and dispose of in accordance with acceptable medical practice and applicable local, state, and federal laws.



This product is single-use only. **Do not reuse. Do not resterilize. Reuse, reprocessing or resterilization of the single-use devices may create a risk of cross contamination, patient infection, and/or malfunction of the device.** After use, dispose of single-use components as infectious waste. Do not use if sterile package is damaged.

STERILE EO



Do not use if sterile package is damaged.



Do not re-use.



Not made with natural rubber latex



Do Not Resterilize

For translations of this instruction, go to [aspensurgical.com/ifu](http://aspensurgical.com/ifu)



Aspen Surgical Products, Inc.  
6945 Southbelt Dr, Caledonia, MI 49316  
phone +1 616.698.7100 toll-free +1 888.364.7004  
[aspensurgical.com](http://aspensurgical.com)

EC REP

Emergo Europe  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

CE  
2797

Made in Mexico  
1-041-1512 Rev 4  
14March2022