Wound Closures

Sterile

aspen surgical.

Rx ONLY

MD

EC REP

Aspen Surgical Products, Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316 USA
Phone +166-698-7100
Toll-Free 888-364-7004
aspensurgical.com

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands
For Country of Origin See Product Label

CE
2797**English****Intended Use**

Intended for the closure of skin lacerations or surgical incisions.

Intended User/ Patient Target Groups

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instruction for Use:

- Skin should be clean, dry, and free of oils to assure good adhesion.
- Device is intended for prolonged contact duration, up to 14 days total with the strips being changed out every 7 days.
- Do not use product if product is damaged.

Warning

Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.

Disposal

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Notice to Users and/or Patients in EU

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Emergo Consulting (UK) Limited