LIGHTED INTUBATION STYLET (Sterile) EN

REF SLOT

INDICATIONS FOR USE:

Sterile Lighted Intubation Styles are used to assist in the proper introduction and positioning of an endotracheal tube (ETT) for the administration of surgical gases. Styles are intended for use in the sterile field of an operating room.

• STORE AT ROOM TEMPERATURE • LIGHT HAS A TYPICAL BURN LIFE OF APPROXIMATELY 2 HOURS

WARNINGS:

Do not use in the presence of flammable gases/materials or oxygen-enriched environments. Fire could result.

Do not extend the stilet tip beyond the distal tip of the tracheal tube. This may result in tissue trauma.

This medical device cannot be effectively cleaned and/or sterilized by the user and therefore cannot be safely reused. It is intended for single use only. Any attempt by the user to clean and resterilize this device may result in bioincompatibility, infection, or other risks of device failure to the patient.

Warning: This product is sterilized utilizing Ethylene Oxide and contains Di(2-ethylhexyl)phthalate(DEHP). These products can expose you to chemicals including Ethylene Oxide (EtO) and Di(2-ethylhexyl)phthalate(DEHP), which is known to the state of California to cause cancer and/or birth defects or other reproductive harm. For more information go to www.P65Warnings.ca.gov.

CAUTIONS:

Do not use if package is damaged or opened.

Do not cut the flexible shaft.

Do not abrade the flexible shaft during insertion or removal of the stylet.

Avoid direct contact with tissue.

Avoid having bulb make contact with hard objects.

IMPORTANT: After use, use proper disposal method.

INSTRUCTIONS FOR USE:

- Select an endotracheal tube (ETT):
 - For SLOT, select an ETT with an inside diameter of no less than 5.5 mm.
 - Apply a water based lubricant to the ETT opening (e.g. lidocaine jelly). Lubrication is essential to advance the ETT without sticking to the stylet.
 - Test the light by activating the switch.
 - Insert the stylet into the ETT until the light source reaches the tip of the ETT. Assure that the distal end of the stylet does not extend beyond the tip of the ETT. When the stylet is in the correct position, slide the EMT clip onto the ETT connector to secure the attachment.
 - Activate the bulb and introduce the ETT per standard practice. When the ETT is in the proper position (evidenced by transillumination of the soft tissues in the patient's neck), remove the stylet, deactivate the light, and dispose of the stylet in a biohazard container.

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