INSTRUCTIONS FOR USE

WriteSite® Plus
Multi-Ink Marking Pen

Non-Sterile

CE

MD

Rx ONLY

EC

REP

Aspen Surgical Products Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316
Phone +1 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

Emergo Europe
Westervoortsewijk 60,
6827 XT Arnhem,
The Netherlands
For Country of Origin See Product Label

English

SKIN MARKER

Intended Use

Skin marker is used for surgical marking on intact skin.

Intended User/ Patient Target Groups

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instructions for Use:

- Works well with chlorhexidine gluconate (CHG) based prep solutions.
- The skin marker ink must be applied to intact, dry skin before applying prep solution. The skin marker ink will not adhere to prep solution or wet skin.
- Prepping the skin with isopropyl alcohol before marking may increase the adherence of the ink to the skin.
- Allow the markings to dry on the skin for approximately 1 minute before rubbing or applying prep solution.
- Apply prep solution as instructed by prep solution manufacturer.
- When prep solution is applied over the skin marker ink, some of the marking will smear, but the skin marker ink will still be visible.
- Rulers packaged with skin markers are not measuring devices and should be used as an indicator only, and not for accurate measurements.
- Not for use on infants.
- Device is intended to write for up to 24 feet (7.32 meters).

Contraindication

Not for ocular use.

Warning

Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.

Disposal

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

UTILITY MARKER

Intended Use

Utility marker is used to provide a permanent method of identifying instruments, specimens, medication labels, and other objects on and off the sterile field.

Intended User/Patient Target Groups

Intended to be used by Healthcare Professionals on instruments or objects in the operating room.

Instructions of Use:

- Utility marker ink is not for use on skin.
- Contains a black, waterproof ink that can withstand autoclave sterilization on most surfaces.
- Mark desired object or surface with the utility marker according to facility protocol.
- Allow markings to dry on object for approximately 1 minute prior to use or autoclave sterilization.
- Device is intended to write for up to 24 feet (7.32 meters).

Disposal

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Sterilization Instructions

To ensure SAL of 10⁻⁶, this product must be sterilized via gamma sterilization method in its original packaging configuration at a sterilization dose range of 28-50 kGy.

Warning

Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.

Notice to Users and/or Patients in EU

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

⚠ WARNING: This product can expose you to chemicals including Gentian violet (Crystal violet), which is known to the State of California to cause cancer. For more information go to www.P65Warnings.ca.gov.

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AB'deki Kullanıcılara ve/veya Hastalara Yönelik Bildirim
Emergo Consulting (UK) Limited