symmetry surgical®

Symmetry Surgical® Skin Markers

Symmetry Surgical Inc.

3034 Owen Drive
Antioch, TN 37013 USA

☎ 1-800-251-3000

Fax: 1-615-964-5566

www.symmetrysurgical.com

symmetry surgical®

Symmetry Surgical GmbH
Maybachstraße 10,
78532 Tuttlingen, Germany

☎ +49 7461 96490

Fax: +49 7461 77921

CE 2797

LCN 205805-001/J

© 2008–2020 Symmetry Surgical Inc.

♻ Revised 07/2020

ENGLISH

IMPORTANT INFORMATION

Please Read Before Use

Symmetry Surgical® Skin Markers

STERILE	R
---------	---

Description

The Symmetry Surgical® Skin Markers are supplied sterile in boxes of 36.

Catalog No.

43-1030

Description

Symmetry Surgical® Skin Marker

43-1032

Symmetry Surgical® Micro Skin Marker

Indications

The Symmetry Surgical® Skin Markers are intended for use as a surgical skin marker on intact skin.

WARNING

The use of this product on surfaces other than intact skin (e.g., exposed internal organs, cornea, or compromised skin surfaces) can result in an adverse reaction.

Warning: This product contains gentian violet which is known to the state of California to cause cancer. For more information go to www.P65Warnings.ca.gov.

Precautions

Inspect the sterile package carefully.

Do not use if:

- the package or seal appears damaged,
- contents appear damaged, or
- the expiry date has passed.

Sterility

The Symmetry Surgical® Skin Markers and Micro Skin Markers are intended for SINGLE USE ONLY; DO NOT RESTERILIZE. Use aseptic technique in all phases of handling. Symmetry Surgical® Single Use devices have not been

designed to undergo or withstand any form of alteration, such as disassembly, cleaning or resterilization, after a single patient use. Reuse can potentially compromise device performance and any usage beyond the design intent of this single-use device may result in unpredictable loss of functionality.

Symmetry Surgical® will not be responsible for any product that is resterilized, nor accept for credit or exchange any product that has been opened but not used.

As long as the individual package is not opened or damaged, the product is sterile.

Warranty

Symmetry Surgical® warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

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