INSTRUCTIONS FOR USE

WriteSite® Plus
Skin Markers
Non-Sterile

aspensurgical

 **MD Rx ONLY**

Aspen Surgical Products Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316
Phone +1 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

EC REP
Emergo Europe
Westervoortsewijk 60,
6827 AT Arnhem,
The Netherlands

For Country of Origin See Product Label

English

Intended Use
Skin marker is used for surgical marking on intact skin.

Intended User/Patient Target Groups
Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instructions for Use:

- Works well with chlorhexidine gluconate (CHG) based prep solutions.
- The skin marker ink must be applied to intact, dry skin before applying prep solution. The skin marker ink will not adhere to prep solution or wet skin.
- Prepping the skin with isopropyl alcohol before marking may increase the adherence of the ink to the skin.
- Allow the markings to dry on the skin for approximately 1 minute before rubbing or applying prep solution.
- Apply prep solution as instructed by prep solution manufacturer.
- When prep solution is applied over the skin marker ink, some of the marking will smear, but the skin marker ink will still be visible.
- Rulers packaged with skin markers are not measuring devices and should be used as an indicator only, and not for accurate measurements.
- Not for use on infants.
- Device is intended to write for up to 24 feet (7.32 meters).

Contraindication
Not for ocular use.

Sterilization Instructions
To ensure SAL of 10x-6, this product must be sterilized via gamma sterilization method in its original packaging configuration at a sterilization dose range of 28-50 kGy.

Warning
Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.

Disposal
Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Notice to Users and/or Patients in EU
Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

⚠ WARNING: This product can expose you to chemicals including Gentian violet (Crystal violet), which is known to the State of California to cause cancer. For more information go to www.P65Warnings.ca.gov.

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AB'deki Kullanıcılara ve/veya Hastalara Yönelik Bildirim
Emergo Consulting (UK) Limited