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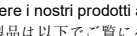
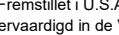
Description:
Olsen Medical® Single Use Tungsten Micro Needle electrological devices are sold as sterile, single use devices, unless otherwise noted. These devices are designed to function as active electrodes in electrological systems.
Indications:
Olsen Medical® Single Use Tungsten Micro Needle Electrodes are intended for use in coagulating and cutting tissue during surgical procedures and are intended for use with a maximum voltage of 5000 Volts (Vpeak).
Handling:
1. Inspect package for damage before use. Discard if damaged or seal is broken. 2. Inspect device after opening in a sterile field. Discard if damaged. 3. Remove and discard the tip protector before use. 4. Device use and application to be determined by physician.
Storage:
These devices should be stored in their original labeled packaging or in protected storage to prevent damage during handling. Take precautions to ensure that the sterile devices are stored in areas where it is completely dry, and protected from extremes of temperature and humidity.
Sterility:
Sterility is guaranteed up to the expiration date unless package is opened or seal is broken.
Sterilization:
Single Use Only (intended to be used on one patient during one procedure). Dispose after use. Do not resterilize.
Caution:
• Inspect every device before use. Damaged electrological devices may cause electrical burns to the patient or physician. • Active electrological devices must be placed away from the patient when not in use. A holster is recommended. • Use a wet sponge to remove eschar build up from the active electrode tips. • Consult RF generator Owner’s Manual. Use only in the active RF generator output connections. • Electromagnetic compatibility (EMC) – Olsen Medical devices are considered medical electrical equipment. Medical electrical equipment may require precautions regarding EMC and may need to be installed or put into service according to the EMC information provided in the operator’s manual of the generator. • Do not use electrological equipment unless properly trained to use it in the specific procedure being undertaken. Use by physicians without such training may result in serious, unintended patient injury. • Do not perform electrological operations in the presence of flammable anesthetics or other flammable gases, fluids or objects, or in the presence of oxidizing agents, as fire could result. • Avoid unnecessary or prolonged activation. Activate only when touching or immediately next to the intended point of contact. • Check that the electrode is compatible with the pen/pencil or handle and ensure that the insulated sleeve fits securely inside the handle. Improperly seated electrodes may cause burn or injury. • Do not bend, cut, or modify electrode before or during surgery. • Keep the voltage/power settings at the lowest possible level to achieve the desired effect. • After use, safely discard in accordance with established procedures for biohazardous waste.
Warning:
Improper use of any element in electrological systems may cause serious burn, shock, and fire or explosion hazard. The use of the device with safely functioning and compatible handles, cords, electrological generators and accessories is the user’s responsibility. Read and understand all precautions and instructions before attempting to use any electrological devices. This product is for SINGLE USE ONLY; DO NOT RESTERILIZE. Use aseptic technique in all phases of handling. Olsen Medical Single Use devices have not been designed to undergo or withstand any form of alteration, such as disassembly, cleaning or resterilization, after a single patient use. Reuse can potentially compromise device performance and any usage beyond the design intent of this single use device may result in unpredictable loss of functionality.

ENGLISH

Description:
Olsen Medical® Single Use Tungsten Micro Needle electrological devices are sold as sterile, single use devices, unless otherwise noted. These devices are designed to function as active electrodes in electrological systems.
Indications:
Olsen Medical® Single Use Tungsten Micro Needle Electrodes are intended for use in coagulating and cutting tissue during surgical procedures and are intended for use with a maximum voltage of 5000 Volts (Vpeak).
Handling:
1. Inspect package for damage before use. Discard if damaged or seal is broken.
2. Inspect device after opening in a sterile field. Discard if damaged.
3. Remove and discard the tip protector before use.
4. Device use and application to be determined by physician.
Storage:
These devices should be stored in their original labeled packaging or in protected storage to prevent damage during handling. Take precautions to ensure that the sterile devices are stored in areas where it is completely dry, and protected from extremes of temperature and humidity.
Sterility:
Sterility is guaranteed up to the expiration date unless package is opened or seal is broken.
Sterilization:
Single Use Only (intended to be used on one patient during one procedure). Dispose after use. Do not resterilize.
Caution:
• Inspect every device before use. Damaged electrological devices may cause electrical burns to the patient or physician.
• Active electrological devices must be placed away from the patient when not in use. A holster is recommended.
• Use a wet sponge to remove eschar build up from the active electrode tips.
• Consult RF generator Owner’s Manual. Use only in the active RF generator output connections.
• Electromagnetic compatibility (EMC) – Olsen Medical devices are considered medical electrical equipment. Medical electrical equipment may require precautions regarding EMC and may need to be installed or put into service according to the EMC information provided in the operator’s manual of the generator.
• Do not use electrological equipment unless properly trained to use it in the specific procedure being undertaken. Use by physicians without such training may result in serious, unintended patient injury.
• Do not perform electrological operations in the presence of flammable anesthetics or other flammable gases, fluids or objects, or in the presence of oxidizing agents, as fire could result.
• Avoid unnecessary or prolonged activation. Activate only when touching or immediately next to the intended point of contact.
• Check that the electrode is compatible with the pen/pencil or handle and ensure that the insulated sleeve fits securely inside the handle. Improperly seated electrodes may cause burn or injury.
• Do not bend, cut, or modify electrode before or during surgery.
• Keep the voltage/power settings at the lowest possible level to achieve the desired effect.
• After use, safely discard in accordance with established procedures for biohazardous waste.

Warning:
Improper use of any element in electrological systems may cause serious burn, shock, and fire or explosion hazard. The use of the device with safely functioning and compatible handles, cords, electrological generators and accessories is the user’s responsibility. Read and understand all precautions and instructions before attempting to use any electrological devices. This product is for SINGLE USE ONLY; DO NOT RESTERILIZE. Use aseptic technique in all phases of handling. Olsen Medical Single Use devices have not been designed to undergo or withstand any form of alteration, such as disassembly, cleaning or resterilization, after a single patient use. Reuse can potentially compromise device performance and any usage beyond the design intent of this single use device may result in unpredictable loss of functionality.
DANSK

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