0020119 Rev. 5
<p>For product information, contact: Yderligere produktoplysninger fås ved at kontakte: Pour obtenir des informations sur le produit, veuillez contacter :</p> <p>Für Produktinformation kontaktieren Sie: Για πληροφορίες σχετικά με το προϊόν, επικοινωνήστε με: Per informazioni sul prodotto, contattate: 製品情報、お問い合わせ先: Para obtener información del producto, póngase en contacto con:</p> <p>symmetry® olsen®</p> <p>3230 Cammeres Center Place Louisville, Kentucky, 40211 USA (502) 813-5022 ⇄ (800) 297-6344 See our products at Se vores produkter på Découvrez nos produits sur Siehe unsere Produkte auf Δείτε τα προϊόντα μας στην ιστοσελίδα È possibile vedere i nostri prodotti all'indirizzo web 当社の製品は以下でご覧になれます Consulte nuestros productos en http://www.symmetryolsen.com</p> <p>CE 1639</p> <p>EC REP</p> <p>Symmetry Surgical GmbH Maybachstraße 10, 78532 Tuttlingen, Germany +49 7461 96490</p> <p>BY ONLY</p> <p>SINGLE USE TUNGSTEN MICRO NEEDLE ELECTRODES ΜΙΚΡΟΝΑΛΕΕΚΤΡΟΔΕΡ ΑΦ ΤΥΓΣΤΕΝ ΤΙΛ ΕΓΓΑΓΣΒΡΥΓ WOLFRAM-MIKRONADELEKTRODEN ZUR EINMALVERWENDUNG ÉLECTRODES À MICRO-AIGUILLE EN TUNGSTÈNE À USAGE UNIQUE ΜΙΑΣ ΧΡΗΣΗΣ ΗΛΕΚΤΡΟΔΙΑ ΜΙΚΡΟΒΕΛΟΝΑΣ ΒΟΛΦΡΑΜΙΟΥ ELETTRODI MICROAGO MONOUSO IN TUNGSTENO 使い捨てタングステン マイクロニードル電極 ELECTRODOS DE MICROAGUJA DE TUNGSTENO DE UN SOLO USO</p>
<p>ENGLISH</p> <p>Description: Olsen Medical® Single Use Tungsten Micro Needle Electrodsurgical devices are sold as sterile, single use devices, unless otherwise noted. These devices are designed to function as active electrodes in electrocsurgical systems. Intended Use (per EU definition) / Indications (per FDA definition): Olsen Medical® Single Use Tungsten Micro Needle Electrodes are intended for use in coagulating and cutting tissue during surgical procedures and are intended for use with a maximum voltage of 5000 Volts (Vpeak). Handling: 1. Inspect package for damage before use. Discard if damaged or seal is broken. 2. Inspect device after opening in a sterile field. Discard if damaged. 3. Remove and discard the tip protector before use. 4. Device use and application to be determined by physician. Storage: These devices should be stored in their original labeled packaging or in protected storage to prevent damage during handling. Take precautions to ensure that the sterile devices are stored in areas where it is completely dry, and protected from extremes of temperature and humidity. Sterility: Sterility is guaranteed up to the expiration date unless package is opened or seal is broken. Sterilization: Single Use Only (intended to be used on one patient during one procedure). Dispose after use. Do not resterilize. Reusing or reprocessing single use devices may cause damage to the device which in turn may cause unnecessary harm to the user and/or patient. Reuse or reprocessing is not recommended. Notices: • Electrocsurgical equipment and accessories are intended to be used by health professional educated in their use. • After Use or upon determining product can no longer be used due to wear or damage, safely discard in accordance with established procedures for biohazardous waste. • Any serious incident that has occurred in relation to the device should be reported to Symmetry Surgical Inc., the governing body, and the Competent authority of the member state in which the incident occurred. • Please refer to the manufacturer of the generator for warnings, precautions, contraindications, undesirable side-effect, measures to be taken, and limitations of use for the electrocsurgical system and accessories. • For applicable IFU/additional information: Visit www.symmetryurgical.com. Caution: • Inspect every device before use. Damaged electrocsurgical devices may cause electrical burns to the patient or physician. • Active electrocsurgical devices must be placed away from the patient when not in use. A holster is recommended. • Use a wet sponge to remove eschar build up from the active electrode tips. • Consult RF generator Owner’s Manual. Use only in the active RF generator output connections. • Electromagnetic compatibility (EMC) – Olsen Medical devices are considered medical electrical equipment. Medical electrical equipment may require precautions regarding EMC and may need to be installed or put into service according to the EMC information provided in the operator’s manual of the generator. • Do not use electrocsurgical equipment unless properly trained to use it in the specific procedure being undertaken. Use by physicians without such training may result in serious, unintended patient injury. • Do not perform electrocsurgery in the presence of flammable anesthetics or other flammable gases, fluids or objects, or in the presence of oxidizing agents, as fire could result. • Avoid unnecessary or prolonged activation. Activate only when touching or immediately next to the intended point of contact. • Check that the electrode is compatible with the pen/pencil or handle and ensure that the insulated sleeve fits securely inside the handle. Improperly seated electrodes may cause burn or injury. • Do not bend, cut, or modify electrode before or during surgery. • Keep the voltage/power settings at the lowest possible level to achieve the desired effect. • After use, safely discard in accordance with established procedures for biohazardous waste. • Electrodes must be compatible with the pen/pencil or handle and ensure that the insulated sleeve fits securely inside the handle with no exposed metal from the electrode shaft. Improperly seated electrode may cause burn or injury. Warning: Improper use of any element in electrocsurgical systems may cause serious burn, shock, and fire or explosion hazard. The use of the device with safely functioning and compatible handles, cords, electrocsurgical generators and accessories is the user’s responsibility. Read and understand all precautions and instructions before attempting to use any electrocsurgical devices. This product is for SINGLE USE ONLY; DO NOT RESTERILIZE. Use aseptic technique in all phases of handling. Olsen Medical Single Use devices have not been designed to undergo or withstand any form of alteration, such as disassembly, cleaning or resterilization, after a single patient use. Reuse can potentially compromise device performance and any usage beyond the design intent of this single use device may result in unpredictable loss of functionality. Warning: Avoid HF output mode/power settings where maximum output voltage may exceed the stated maximum voltage. Warning: Do not make any modification to these devices. Modification may cause burn or injury. Warning: Due to concerns about the carcinogenic and infectious potential of electrocsurgical byproducts (such as tissue smoke plume and aerosols which may include toxic gasses and vapors, live and dead cellular material, and viruses), protective eyewear, filtration masks, and effective smoke evacuation equipment should be used in both open and laparoscopic procedures. Contact Symmetry Surgical at customerservice@symmetryurgical.com for additional information or to inquire about our smoke evacuation solutions.</p>
<p>DANSK - DANISH</p> <p>Beskrivelse: Olsen Medical® Mikronåleelektroder af tungsten til engangsbrug sælges som sterile instrumenter til engangsbrug, medmindre andet er angivet. Disse instrumenter er designet til at virke som aktive elektroder i elektrokirurgiske systemer. Tilslaget brug (i henhold til EU-definition/Indikatorer (i henhold til FDA-definition): Olsen Medical® Mikronåleelektroder af tungsten til engangsbrug er beregnet til brug ved vævskoagulering og -skæring under kirurgiske indgreb og er beregnet til brug med en maksimal spænding på 5 000 Volt (Vpeak). Håndtering:</p>

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