INSTRUCTIONS FOR USE RETURN ELECTRODE PLATE EN

REF A1204, A1204P, A802EU, A802

WARNINGS AND PRECAUTIONS

If the patient is repositioned for the surgical procedure, verify that the plate remains in proper contact with the skin and the cable remains connected to the plate. Failure to do so may result in patient burns.

During the surgical procedure, assure that no object obstructs the plate site.

If higher than normal power settings are required, a problem may exist. Before increasing the power setting, verify that the plate has full contact with patient's skin. Check the cable and connectors and inspect all active accessories.

Do not relocate the plate after initial application.

Clean the plate with a cleaning solution. (DO NOT AUTOCLAVE.)

Do not cut or modify the plate in any way.

Review and follow any additional instructions, warnings, and precautions in the Operating Instructions for the generator in use.

Selection and Preparation of Placement Site

1. Select a well-vascularized, muscular, and convex site in proximity to the surgical site. Do not place over scar, inflamed skin, fatty tissue, bony prominences, metal prostheses, ECG electrodes and cables, pacemakers, or where fluids may pool.
2. Shave all hair from the selected site. Dry skin thoroughly.

Placement of Electrode Plate


1. Inspect the cable for visible damage prior to use. Do not use a cable that has cuts, nicks, or missing insulation.
2. Connect the electrode plate to the cable by holding the cable's connector in place and inserting the plate into the connector.

For plates with a twist-on connector, insert the plate into the connector and rotate the plate clockwise. (DO NOT ROTATE THE CONNECTOR AS DAMAGE TO THE CORD MAY OCCUR.) See Figure 1.

3. Locate electrode plate away from any metal objects or metal operating room tables.
4. When using A1204, A1204P, A802EU or A802, use conductive gel to make contact between the patient's skin and the electrode plate.

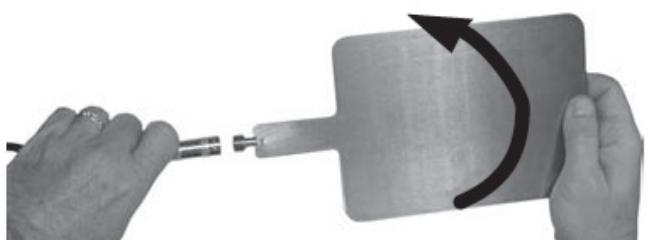FIGURE 1 (A1204, A1204P, A802EU, A802)

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