# O'Sullivan-O'Connor Abdominal Retractor

**Symmetry Surgical Inc.**  
3034 Owen Drive,  
Antioch, TN 37013 USA  
☎ 1-800-251-3000  
Fax: 1-615-964-5566  
[www.symmetrysurgical.com](http://www.symmetrysurgical.com)

Symmetry Surgical GmbH  
Maybachstraße 10  
78532 Tuttlingen, Germany  
☎ + 49 7461 96490  
Fax: + 49 7461 77921

**CE 2797**

**LCN-500-4400-85/M**

© 2002–2020 Symmetry Surgical  
♻ Revised 08/2020

# ENGLISH

## IMPORTANT INFORMATION

Please Read Before Use

### O'Sullivan-O'Connor Abdominal Retractor

#### Description

The O'Sullivan-O'Connor Abdominal Retractor (catalog no. 50-4400) is a self-retaining instrument that includes two small blades and one large blade.

#### WARNINGS

Read this section carefully before using the retractor to avoid femoral nerve compression injury.

1. Wherever possible, avoid using this retractor during tubal reconstruction, but particularly in thin patients and in conjunction with the Pfannenstiel incision.
2. When positioning this retractor, carefully check the relationship of the lateral blade to the area of the nerve WITH *downward* and *outward* pressure on the retractor. This is especially important if the patient is tilted toward the surgeon.
3. Use extreme care not to apply pressure to the retractor during surgery. Use a specially designed armrest for stabilization of the arm as required.

#### Reprocessing

Refer to Symmetry Surgical Instructions for Use 204233 for updated information regarding reprocessing the instrument.

Copies of this instruction sheet are available by requesting IFU 204233 from Symmetry Surgical Customer Service at:

Telephone: 800 251 3000  
Fax: +1 615 964 5566  
E-mail: ssi-customerservice@symmetrysurgical.com

### **CAUTION:**

It is extremely important that movable parts be properly lubricated to keep them functional. It is recommended that all components be immersed in an instrument lubricant such as PRESERVE® Concentrate (catalog no. 43-1033).

### **Service and Repair**

For service or repair outside the United States, contact your local Symmetry Surgical® representative.

Inside the United States, send instruments for service or repair to:

Symmetry Surgical  
3034 Owen Drive,  
Antioch, TN 37013 USA

Always include a repair purchase order number and a written description of the problem.

### **Warranty**

Symmetry Surgical® warrants that this medical device is free from defects in both materials and workmanship. **Any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Suitability for use of this medical device for any particular surgical procedure should be determined by the user in conformance with the manufacturer's instructions for use. There are no warranties that extend beyond the description on the face hereof.**

PRESERVE® is a registered trademark of Symmetry Surgical

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Fax : +1 615 964 5566

E-mail :

ssi-customerservice@symmetrysurgical.com

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