Bovie® SF35 Filter Instructions

Distributed By:

Symmetry Surgical Inc.

3034 Owen Drive
Antioch, TN 37013 USA
1-800-251-3000
SRN: US-MF-000024174

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug
Switzerland

MC-23100 r3
2024-03-31

R_x ONLY

Bovie® SF35 Filter Instructions

Note: Before installing or removing any filter, be sure that the system is turned off.

Filter Installation Instructions:

WARNING: This device is not intended for evacuation of fluid. If fluid is expected to be aspirated using the Bovie® SF35 Filter or the Bovie® Smoke Shark II system, fluid collection devices must be installed with the vacuum hose assembly. Failure to install a fluid collection device may cause filter blockage and/or electrical damage.

The installation of the Bovie® SF35 Filter into the Bovie® Smoke Shark II Smoke Evacuation System(s) is quick and simple.

1. Remove the Bovie® SF35 Filter from the shipping box and discard any protective wrapping. Examine all filters for damage during shipping and storage. Do not install any filter with visible signs of structural damage.
2. Insert the Bovie® SF35 Filter into the filter receptacle. Be sure that the filter is seated completely against the bottom of the filter chamber and clip is fully engaged.

Filter Removal Instructions:

WARNING: The Bovie® SF35 Filter should be changed when the Filter Life Indicator shows red flashing LED (replace). Failure to change this filter may result in decreased efficiency and contamination of the electric motor, vacuum pump, and sound absorbing media within the system, or non-operation of smoke evacuator.

CAUTION: Using any other filter or accessory not supplied by Bovie Medical Corporation may cause damage to the system and/or cause the system to be inoperable and may void the warranty.

1. After the Bovie® SF35 Filter has been exhausted and requires changing, turn the smoke evacuation system off and disconnect any accessory tubing attached to the filter.
2. Depress the filter tab and pull the filter from the smoke evacuation system and dispose of in accordance with hospital policy. The Bovie® SF35 Filter may be disposed of or incinerated.
3. Wipe unit with a damp cloth containing mild disinfectant solution or soapy water. Wipe dry with a clean cloth. Do not steam sterilize. Follow the indicated instructions for maintenance and installation of a new Bovie® SF35 Filter.

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