EYE BUBBLE EN
[REF] 0002
STORE AT ROOM TEMPERATURE
WARNING: This medical device cannot be effectively cleaned and/or sterilized by the user and therefore cannot be safely reused. It is intended for single use only. Any attempt by the user to clean and resterilize this device may result in bioincompatibility, infection, or other risks of device failure to the patient.

 WARNING: This product is sterilized utilizing Ethylene Oxide. These products can expose you to chemicals including Ethylene Oxide (EtO), which is known to the state of California to cause cancer and/or birth defects or other reproductive harm. For more information go to www.P65Warnings.ca.gov.
--

IMPORTANT: Remove adhesive backing prior to use.

IMPORTANT: To ensure secure adhesion, properly prepare application site.

Medical purpose/indication Bovie® Eye Bubbles are indicated for use after ophthalmic procedures where eye protection may be required. The eye bubble may also be used to protect the eye during other surgical procedures.
Operating Principle The eye bubble provides a plastic protective barrier between the patient eye and environment.
Patient Population No Restrictions
Site of Use and Site Condition Site of use — Eye Site Condition — Aseptic
Intended User Profile This device is intended to be used by trained physicians.
Education – Restricted for use by or on the order of physicians only
Knowledge – Read and understands the supplied IFU – Understands hygiene
Experience – Some training on techniques or training under surveillance/supervision
Permissible impairments – Mild reading vision impairment or corrected vision to 20/20
Intended Conditions for Use – Doctor Office, Surgery center or Hospital, intended for professional use only – Aseptic site; Sterile medical device – Lit, Clear and unobstructed view at point of use – Ambient Temperature 10° C to 40° C
Frequency of Use Single use – Single patient
Disposal – Biohazard – Dispose of device according to facility procedures
Mobility – Hand-Held, portable

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STORAGE
10 to 75 % non-condensing

OPERATE
30 to 75 % non-condensing

TRANSPORT
10 to 100 % including condensation

Operationsprinzip Die Augenblase bietet eine Plastikschutzbarriere zwischen dem Auge des Patienten und der Umgebung.	Utilização única - Um único paciente Eliminação – Perigo biológico – Elimine o dispositivo de acordo com os procedimentos da instituição
---	--

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