



Virtual Classrooms with Anniewhere

Here is a quick-guide to help you setup your virtual classroom using Anniewhere

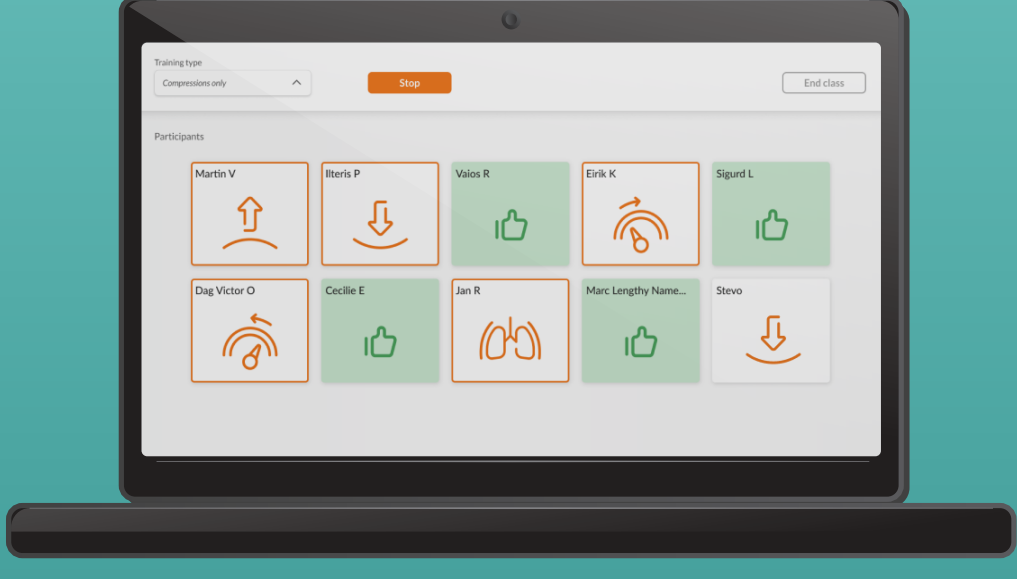

&

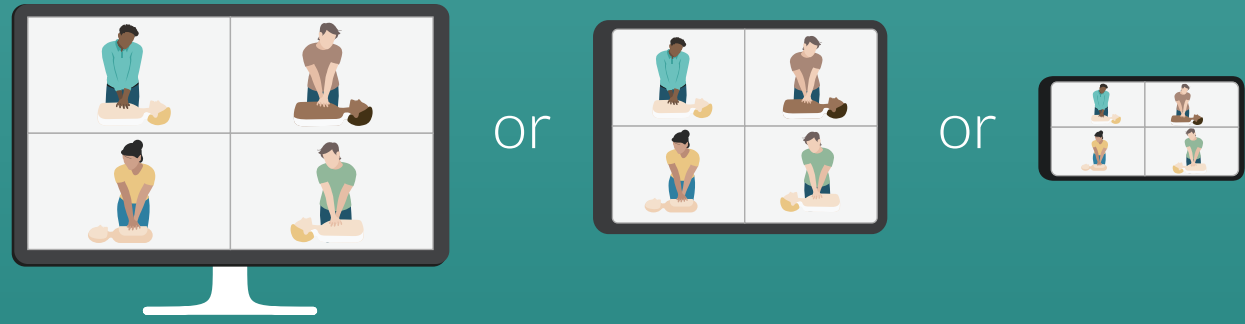

1

Get ready for your virtual classes

Access Anniewhere on any device and start training quality lifesavers from anywhere.

It is recommended for both instructor and learner to use an extra screen or another device for your video conference during the class.

2




Start a class


Click 'Start a Class' on the Anniewhere homepage and share the 'Classroom Code' with the learners.

The Learners join a course by entering the code and their name into the QCPR Classroom App.

You are able to see who has successfully joined the class once learners start clicking 'Join Class'. Click 'Continue' once everybody has joined.

Learner Identification:
Remind learners to type name as it should appear on the course completion card or certificate.

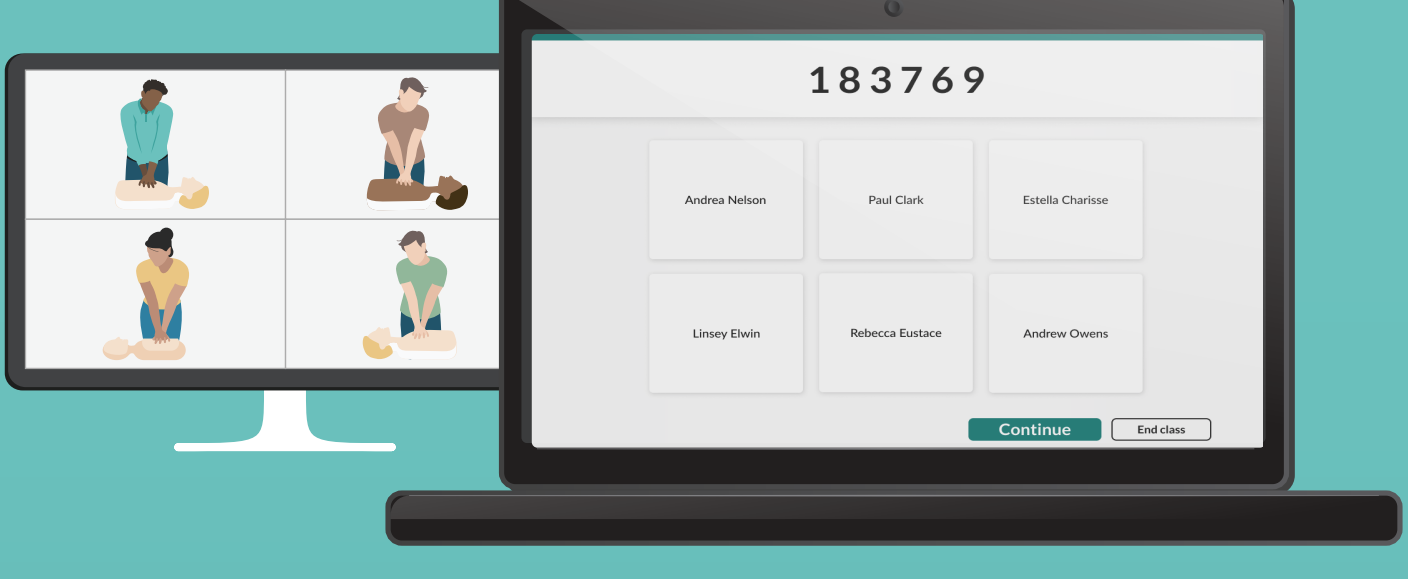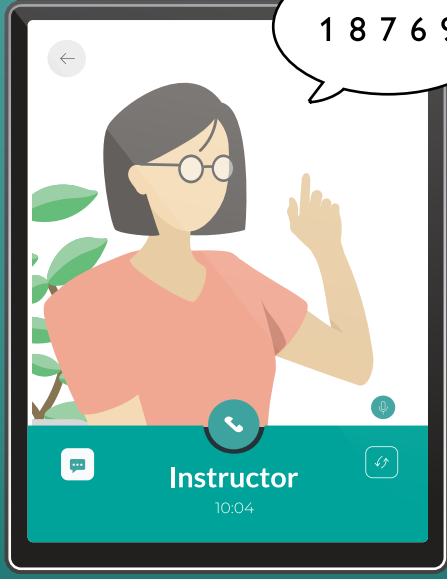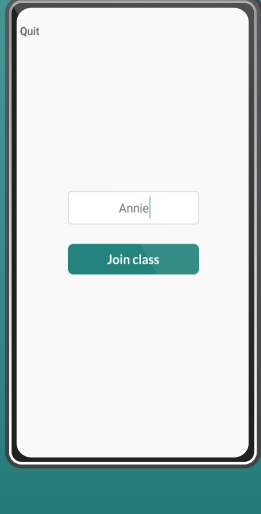

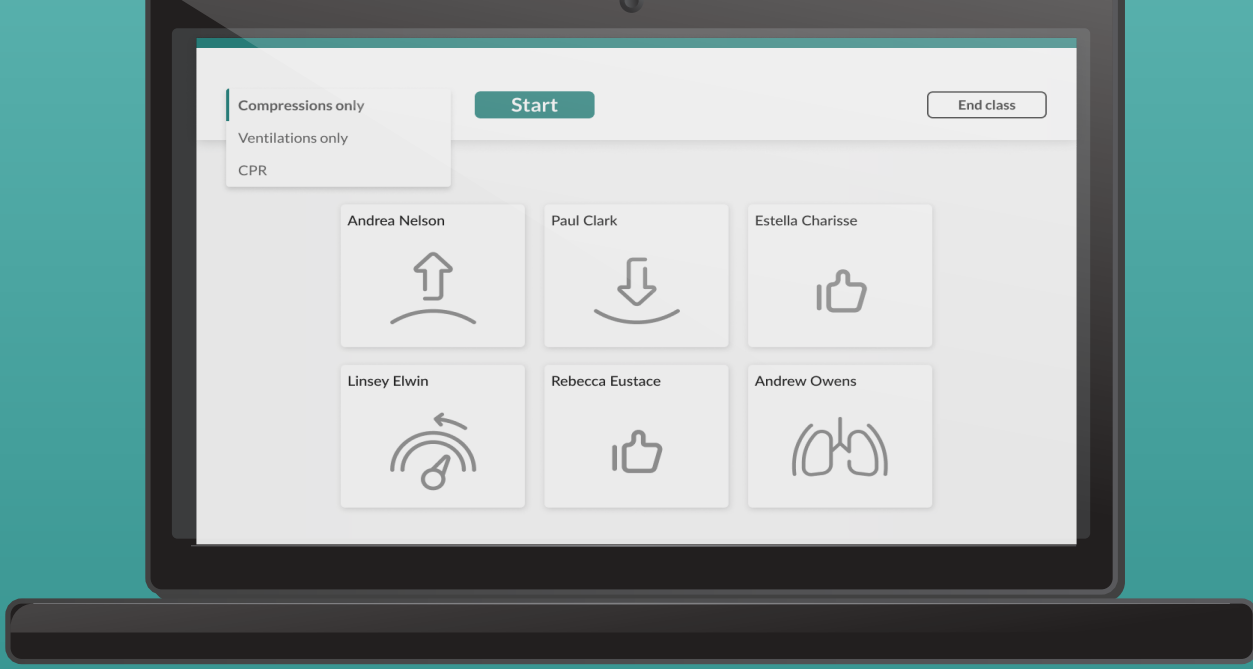3

Start a training session

Select a 'Session type' from the dropdown menu on the top-left corner of your screen and click 'Start'. You can keep track of the timer and stop a session at any given time by clicking 'Stop'.

Available session types:
Compressions only, Ventilations only or CPR (30:2)

Laerdal QCPR manikin use:
Learners compress the manikin to activate and start receiving feedback.


Other manikin use:
Learners use video compression rate feedback.

Learners with activated manikins appear with gray feedback icon in the lobby.


4

Live Feedback


During the session, learners see their individual feedback on their QCPR Classroom app. At the same time, you can observe their skills development via the objective feedback overview and provide feedback to both the learner and the entire class.

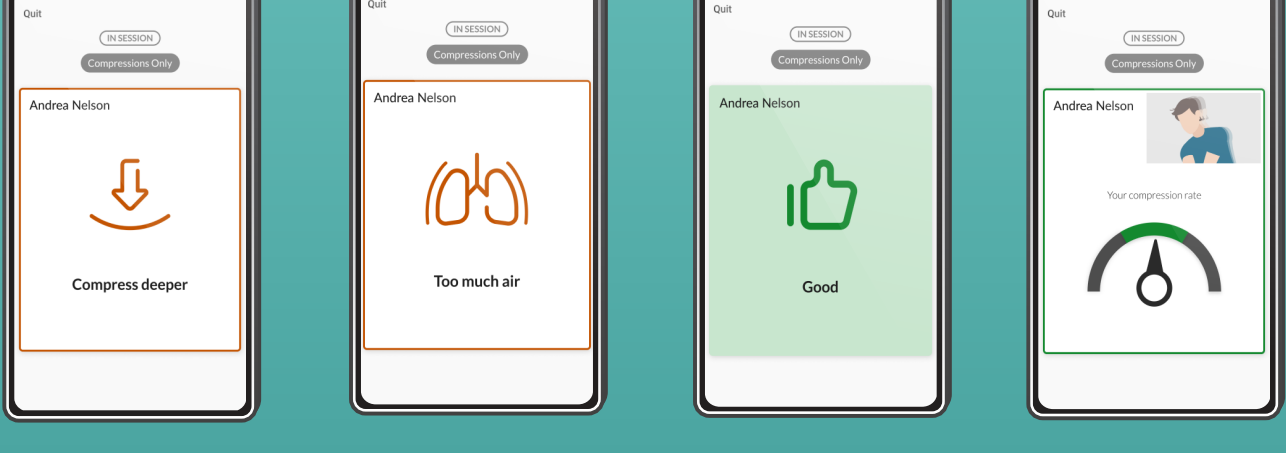

Learner screen

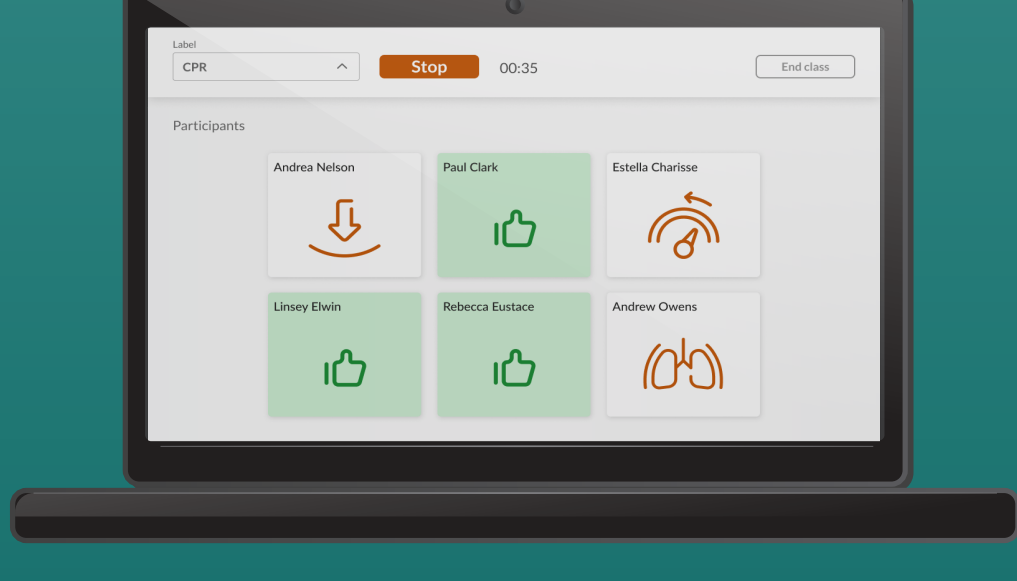

Instructor screen

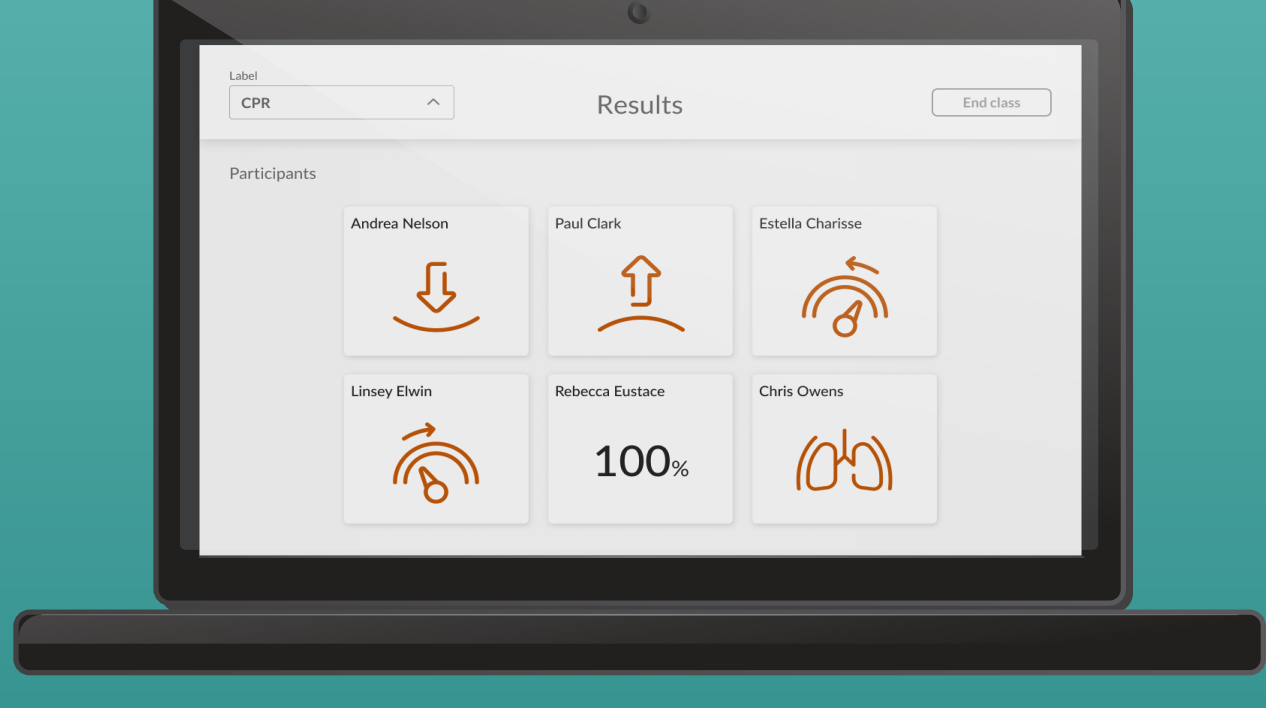


5

Debrief & Results


Following a session, you can go through the results of every learner. To improve the training objectives, you can also provide detailed feedback to the learners by using the score and the improvement tip displayed in the results.

What does each feedback icon mean?


How to interpret the improvement tip on the CPR performance of learners.




Good




Release fully




Compress deeper



Compress faster



Compress slower




Too much air

Tips for the video call

Here are some helpful tips for you to help you conduct a virtual class using Anniewhere with a video call app.

Adjust the camera during video call

When instructors demonstrate to learners

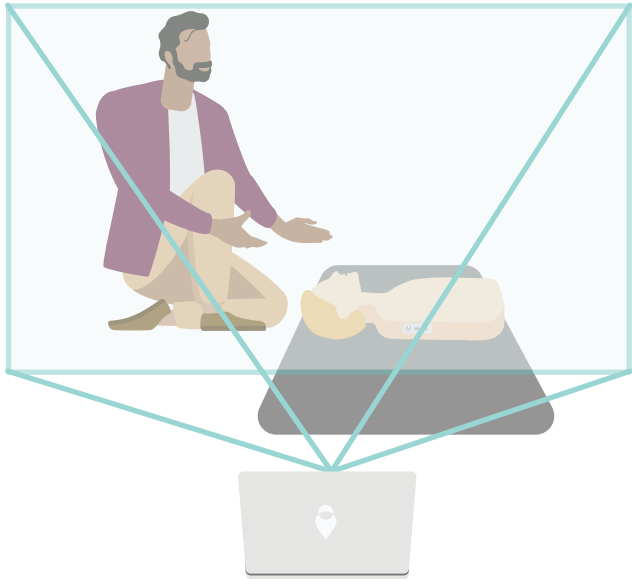

Place yourself and the manikin within the video frame
During the skills demonstration, make sure to place the device you are using for the video call and the manikin on a flat surface and make sure that you and the manikin are visible in the video frame.

Enable/disable clicker

Clickers can be enabled or disabled via the push pull handle under Little Anne QCPR and Little Junior QCPR manikins

Mute/unmute participants

Keep in mind that there might be challenges with hearing individuals during classes with many participants. Get comfortable muting and unmuting participants on your video call app.

Wearing a headset during CPR practice

It can be challenging to maintain the headset during compressions and especially ventilations. If headsets are needed and can not stay in place, it's best to place them on the neck or elsewhere during skills sessions.